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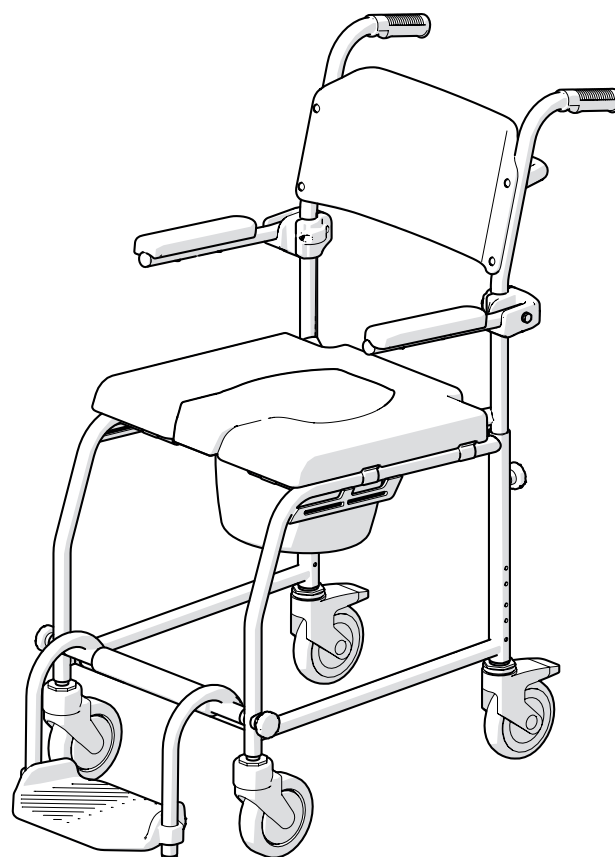
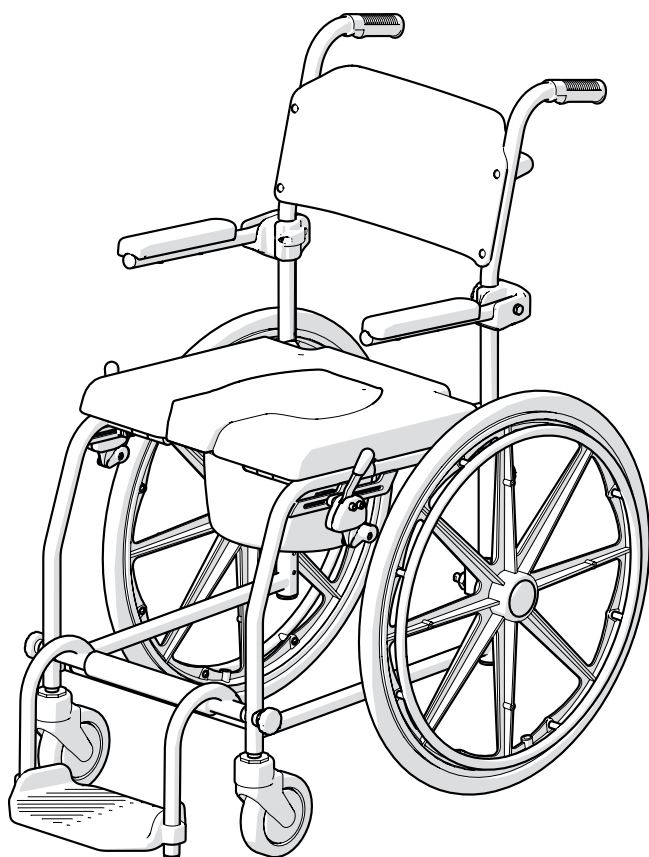
If you are visually impaired, this document can be viewed in pdf format at www.Sunrisemedical.co.uk or alternatively is available on request in large text.

EN 23

Wheelchair Components

We at SUNRISE MEDICAL have been awarded the ISO-13485 certificate, which affirms the quality of our products at every stage, from R & D to production. This product complies with the standards set forth in EU and UK regulations. Options or accessories shown are available at extra cost.

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Luftdruck 795 – 1060 hPa

Torque

Unless otherwise advised, the torque for M6 screws is 7 Nm.

IMPORTANT!

Self locking nuts are used for several connections. If these are loosened, they must be always replaced.

Contents


Assembly Instructions	6
1.0 Definitions	23
2.0 Foreword	24
3.0 Use	25
4.0 Area of Application	25
5.0 General safety notes and driving limits	27
6.0 Warranty	29
7.0 Transportability	30
8.0 Delivery	30
9.0 Handling	31
10.0 Brakes	31
11.0 Footboard	32
12.0 Armrests	32
13.0 Seat	32
14.0 Toilet bucket	32
15.0 Rear wheels	33
16.0 Anti-Tip Tubes	33
17.0 Step Tubes	33
18.0 Safety bar	34
19.0 Transit wheels	34
20.0 Crutch holder	34
21.0 Name Plate	34
22.0 Maintenance and care	35
23.0 Trouble shooting	36
24.0 Disposal / recycling of materials	37
25.0 Technical Specifications	37

NOTE!

The wheelchairs shown and described in this user guide may not correspond in every detail exactly to your own model. However, all instructions are completely relevant, regardless of possible detail differences.

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate, and do not constitute specifications.

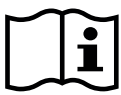1.0 Definitions

Word	Definition
 DANGER!	Advice to the user of Potential Risk of serious injury or death if the advice is not followed
 WARNING!	Advice to the user of a potential risk of injury if the advice is not followed
 CAUTION!	Advice to user that potential damage to equipment may occur if the advice is not followed
 NOTE:	General advice or best practice
	Reference To Additional Documentation
	Does not fit

NOTE:

Please keep a note of your local service agent's address and telephone number in the space provided.

In the event of a breakdown, contact them and try to give all relevant details so they can help you quickly.

Dealer signature and stamp

2.0 Foreword

Dear Customer,

We are very pleased that you have decided in favour of a high-quality SUNRISE MEDICAL product.

This Owner's manual will provide numerous tips and ideas so that your new wheelchair can become a trustworthy and reliable partner in your life.

For Sunrise Medical, it is very important that we have a good relationship with our customers. We like to keep you up-to-date about new and current developments at our company. Keeping close to our customers means: fast service, as little red tape as possible, working closely with customers. When you need replacement parts or accessories, or if you just have a question about your wheelchair – we are there for you.

We want you to be satisfied with our products and service. At Sunrise Medical we are constantly working to develop our products further. For this reason, changes can occur in our palette of products with regard to form, technology, and equipment. Consequently, no claims can be construed from the data or pictures contained in this Owner's manual.

The management system of SUNRISE MEDICAL is certified to EN ISO 13485 and ISO 14001.

 As the manufacturer, SUNRISE MEDICAL, declares that this product conforms to the Medical Device Regulation (2017/745).

 As the manufacturer, SUNRISE MEDICAL, declares that the product conforms to the UK Medical Devices Regulation 2002 No. 618.

Notice to the user and/or patient: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

B4Me special adaptations

Sunrise Medical strongly recommends that in order to ensure that your B4Me product operates, and performs as intended by the manufacturer; all the user information supplied with your B4Me product is read and understood, before the product is first used.

Sunrise Medical also recommends that the user information is not discarded after reading it, but it is kept safely stored for future reference.

Medical Device Combinations

It may be possible to combine this Medical device with one or more other Medical Device or other product. Information on which combinations are possible can be found at www.Sunrisemedical.co.uk. All combinations listed have been validated to meet the General Safety and Performance Requirements, Annex I Nr. 14.1 of the Medical Device Regulation 2017/745.

Guidance on the combination, such as mounting, can be found at www.SunriseMedical.co.uk.

Please contact your local, authorized SUNRISE MEDICAL dealer if you have any questions regarding the use, maintenance or safety of your wheelchair.

In case there is no authorized dealer in your area, or you have any questions about product safety and product recalls, contact Sunrise Medical either in writing or by telephone or find the information on www.sunrisemedical.co.uk.

Sunrise Medical
Thorns Road
Brierley Hill
West Midlands
DY5 2LD
Phone: 0845 605 66 88
Fax: 0845 605 66 89
www.SunriseMedical.co.uk

DO NOT USE YOUR WHEELCHAIR UNTIL THIS MANUAL HAS BEEN READ AND UNDERSTOOD.

3.0 Use

The shower and toilet commode is designed exclusively as an aid for showering or bathing, going to the toilet or for use during personal hygiene or transferring patient.

Shower and toilet commodes are exclusively for adults who are unable to walk or have limited mobility, for their own personal use, when self-propelling and being moved by a third party (pushed by attendant), within the home.

The shower and toilet commode can be used over a standard toilet or independently with the commode pan (optional). It may only be used as a bathing chair in a pool, if the pool does not contain salt water and an attendant is present at all times.

Do not use for any other purpose.

The maximum weight limit (includes the user and any weight of accessories fitted to the wheelchair) is marked on the serial number label, which is affixed to the crossbar or stabilizer bar below the seat.

Warranty can only be taken on if the product is used under the specified conditions and for the intended purposes.

The expected life of the wheelchair is 5 years provided it is used in strict accordance with the intended use as set out in this document and all maintenance and service requirements are met.

Please DO NOT use or fit any 3rd party components to the wheelchair unless they are officially approved by Sunrise Medical.

WARNING

Risk of injury

Never use the product as an aid for getting into or out of a bathtub, for climbing or descending, as a support or for similar purposes.

Do not use the product on surfaces that are not firm (e.g. on loose gravel, in sand, mud, snow, ice or grass) or uneven ground.

DANGER!

Do not fit any unauthorized electronic equipment.

Do not fit powered or mechanically operated mobility devices, drives, or any other device that changes the intended use or the structure of the wheelchair.

Any combination with other medical devices requires approval directly from Sunrise Medical.

4.0 Area of Application

The shower and toilet commode can be used by those who cannot walk or have limited mobility because of:

Indications

The varieties of fitting variants, as well as the modular design, mean that it can be used by those who cannot walk or have limited mobility because of:

- Paralysis
- Loss of extremity (leg amputation)
- Extremity defect/deformity
- Joint contractures/joint injuries
- Illnesses such as heart and circulation deficiencies, disturbance of equilibrium or cachexia, Neurological disease patterns, Muscle dystrophy, Hemiplegia as well as for elderly people who still have strength in the upper body.

Contraindications

The wheelchair shall not be used without attendant in case of:

- Perception disorder
- Imbalance
- Loss of both arms, if not supported by a caregiver
- Joint contracture or joint damage on both arms
- Seating disability

When considering provision, please also note the body size, weight, physical and psychological constitution, the age of the person, living conditions and environment.

NOTE:

Please note that driving a wheelchair requires sufficient cognitive, physical and visual skills. The user must be able to assess the effects of actions during the operation of the wheelchair and, if necessary, to correct them. These capabilities and the safe use of the additionally attached components cannot be assessed by Sunrise Medical as a manufacturer. We cannot accept any liability for any damage resulting from this.

Please refer to the operating instructions of the wheelchair and the additionally mounted components. Instruct the user in the safe use of the wheelchair and the additionally mounted components. Inform users of specific warnings that need to be read, understood, and respected.

Operating Conditions

The shower and toilet commode can be used for institutional or private household indoors only in self-propelled or attendant configurations, on firm, relatively smooth surfaces.

It is not suited for permanent use in a salty atmosphere such as e.g. saline baths. To prevent damages the shower and toilet commode should be cleaned thoroughly with regular tap water after such a use.

Temperature for use: 10 - 65 °C

5.0 General safety notes and driving limits

The engineering and construction of this shower and toilet commode has been designed to provide maximum safety. International safety standards currently in force have either been fulfilled or exceeded in parts. Nevertheless, users and attendants may put themselves at risk by improperly using their wheelchairs. For your own safety, the following rules must be strictly observed.

Unprofessional or erroneous changes or adjustments increase the risk of accident.

Be careful during your first ride in this wheelchair. Get to know your wheelchair.

Before each use, the following should be checked:

- Quick-release axles on the rear wheels
- Tyres, tyre pressure and wheel locks.

Before changing any of the adjustments of this wheelchair, it is important to read the corresponding section of the user's manual.

When riding forwards over a step or up an incline, always approach it in a right angle (90°) and the body should be leaning forward.

Do not use the wheelchair on slopes $> 10^\circ$. The Dynamic safe slope is dependent on the chair configuration, the user's abilities and the style of riding. As the user's abilities and style of riding cannot be pre-determined then the max safe slope cannot be determined. Therefore this must be determined by the user with the assistance of an attendant to prevent tipping. It is strongly recommended that inexperienced users have anti-tip tubes fitted.

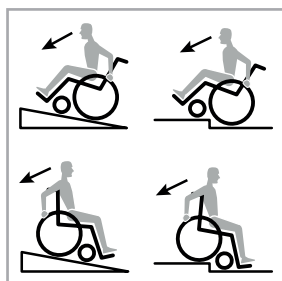
To avoid hand injuries do not grab in between the spokes or between the rear wheel and wheel lock when driving the wheelchair.

Riding sideways on to a slope or incline, increases the possibility of the wheelchair tipping over sideways.

To overcome steps the wheelchair must be tipped and pushed, never carried, over the step. In general, any anti-tip tubes fitted must be set beforehand, so that they cannot touch the steps, as otherwise this could lead to a serious tumble.

DANGER!

NEVER exceed the maximum user weight of 130 kg, for driver plus any items carried on the wheelchair. If you exceed the maximum load, this can lead to damage to the chair, or you may fall or tip over, lose control and may lead to serious injury of the user and other people.

When getting on or off the wheelchair, do not use the footboards. These should be flipped up beforehand. Always position yourself as close as possible to the place where you wish to sit.

Only use your wheelchair properly. For example, avoid

travelling against an obstacle without braking (step, kerb edge) or dropping down gaps.

The wheel locks are not intended to brake your wheelchair. They are only there to ensure that your wheelchair does not begin rolling unintentionally. Always apply both wheel locks.

Lean your upper body further forward when going up slopes and steps.

Lean your upper body further back when going down slopes and steps. Never try to climb and descend a slope diagonally.

Avoid using an escalator which may lead to serious injury in the event of a fall.

Attending person

The attendant must be made aware of all possible dangerous situations before starting any supportive involvement.

Afterwards the anti-tip tubes must be set back to their correct position.

Make sure that the attendant only holds the wheelchair using securely mounted parts (e.g. not on the footrests or the armrests).

This wheelchair is not designed to be used whilst weight training and/or when using dumbbells. Only use equipment specifically designed for this purpose.

Only use the shower and toilet chair for short distances. Health can be affected from long term use.

Do not lift or carry the wheelchair by the backtubes or the pushhandles.

When using the lifting ramp make sure that the anti-tip tubes fitted are positioned outside the danger area.

Anti-tip tubes should prevent the chair tipping over backwards unintentionally.

Under no circumstances should they take the place of transit wheels, and be used to transport a person in a wheelchair with the rear wheels removed.

When reaching for objects (which are in front of, to the side or behind the wheelchair) make sure that you do not lean too far out of the wheelchair, as if you change the center of gravity there is a risk of tipping or rolling over. The hanging of additional load (back pack or similar items) onto your chair backposts can affect the rearward stability of your chair. This can cause the chair to tip backwards causing injury.

Adjustments to your wheelchair, particularly to safety-related components must be carried out by an approved dealer. This applies to adjustments to wheel locks, anti-tip tubes and lower leg length

Do not fit any unauthorized electronic equipment, powered or mechanically operated mobility drives, hand-bike or any other device that changes the intended use or the structure of the wheelchair.

Any combination with other medical devices requires the approval of Sunrise Medical.

Further information and safety instructions can be provided by your authorized dealer.

For thigh amputees you must use anti-tip tubes.

If the seat, back or wheels are damaged, you must replace them immediately.

Be careful with fire, in particular with burning cigarettes. Seat and back can be set alight.

If the wheelchair is subject to direct sunlight for a long period of time, then parts of the wheelchair (e.g. frame, legrests, brakes and sideguard) may become hot (>41°C).

Always make sure that the quick-release axles on the rear wheels are set properly and lock in. If the button on the quick-release axle is not pressed in, the rear wheel cannot be removed.

WARNING!

Always be careful with your fingers when working or adjusting the wheelchair!

WARNING!

CHOKING HAZARD

This mobility aid uses small parts which under certain circumstances may present a choking hazard to small children.

THIS GUARANTEE DOES NOT AFFECT YOUR LEGAL RIGHTS IN ANY WAY.

Sunrise Medical* provides a guarantee, as set out in the warranty conditions, for wheelchairs to its customers covering the following.

Warranty conditions:

1. Should a part or parts of the wheelchair require repair or replacement as a result of a manufacturing and/or material fault within 24 months or for frame and cross-braces within 5 years after delivery to the customer, then the affected part or parts will be repaired or replaced free of charge. The warranty will only cover manufacturing defects
2. To enforce the warranty, please contact the supplier of your wheelchair – e.g. the Sunrise Medical Approved dealership or Healthcare provider with the exact details of the nature of the difficulty. Should you be using the wheelchair outside the area covered by the Sunrise Medical customer service agent, repairs or replacement will be carried out by another agency as designated by the manufacturer. The wheelchair must be repaired by a Sunrise Medical designated Customer Service agent, (dealer).
3. For parts, which have been repaired or exchanged within the scope of this warranty, we provide a warranty in accordance with these warranty conditions for the remaining warranty period for the wheelchair in accordance with point 1).
4. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, (following the fitting), in accordance with these warranty conditions.
5. Claims from this warranty shall not arise, if a repair or replacement of a wheelchair or a part is required for the following reasons:
 - a. Normal wear and tear, which include batteries, armrest pads, upholstery, tyres, brakes shoes, etc.
 - b. Any overloading of the product, please check the product label for maximum user weight.
 - c. The product or part has not been maintained or serviced in accordance with the manufacturer's recommendations as shown in the user instructions and/or the service instructions.
 - d. Accessories have been used which are not specified as original accessories.
 - e. The wheelchair or part having been damaged by neglect, accident or improper use.
 - f. Changes/modifications have been made to the wheelchair or parts, which deviate from the manufacturer's specifications.
 - g. Repairs have been carried out, before our Customer Service has been informed of the circumstances.
6. This guarantee is subject to the law of the country in which the product was purchased from Sunrise Medical*

* Means the Sunrise Medical facility from which the product was purchased.

Additional Notes For Australia Only:

- i. For goods provided by Sunrise medical Pty Ltd in Australia, our goods come with a guarantee by Sunrise Medical that cannot be excluded under Australian Consumer Law.
- ii. You are entitled to a replacement or refund for a major failure and for compensation for any foreseeable loss or damage.
- iii. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.
- iv. The benefits to you given by this warranty are in addition to your other rights and remedies under a law in relation to the goods to which the warranty relates.

7.0 Transportability

The shower and toilet commode cannot be used as a seat in a car.

When transporting the unoccupied wheelchair in a vehicle, it should be tied down or strapped in.

The product can be dismantled for transport into smaller parts. Please see the Assembly section on pages 6-7.

IMPORTANT!

Risk of damage to the product

- Transport the product in a dry area.
- Do not expose the product to direct sunlight.

8.0 Delivery

Your new shower and toilet commode is delivered in a cardboard box. In order to avoid any damage during transportation, loose parts or parts to be fitted, are packed separately in the cardboard box.

Unpacking:

Check the shipping cardboard box for external damage which may have been caused during transportation.

Remove all packaging material.

Take the Delphin carefully out of the box and check that the delivery is complete.

Check all parts for surface damage, scratches, cracks, dents, distortions and other defects.

NOTE: Should you find any damage, or if any parts are missing please let us know immediately.

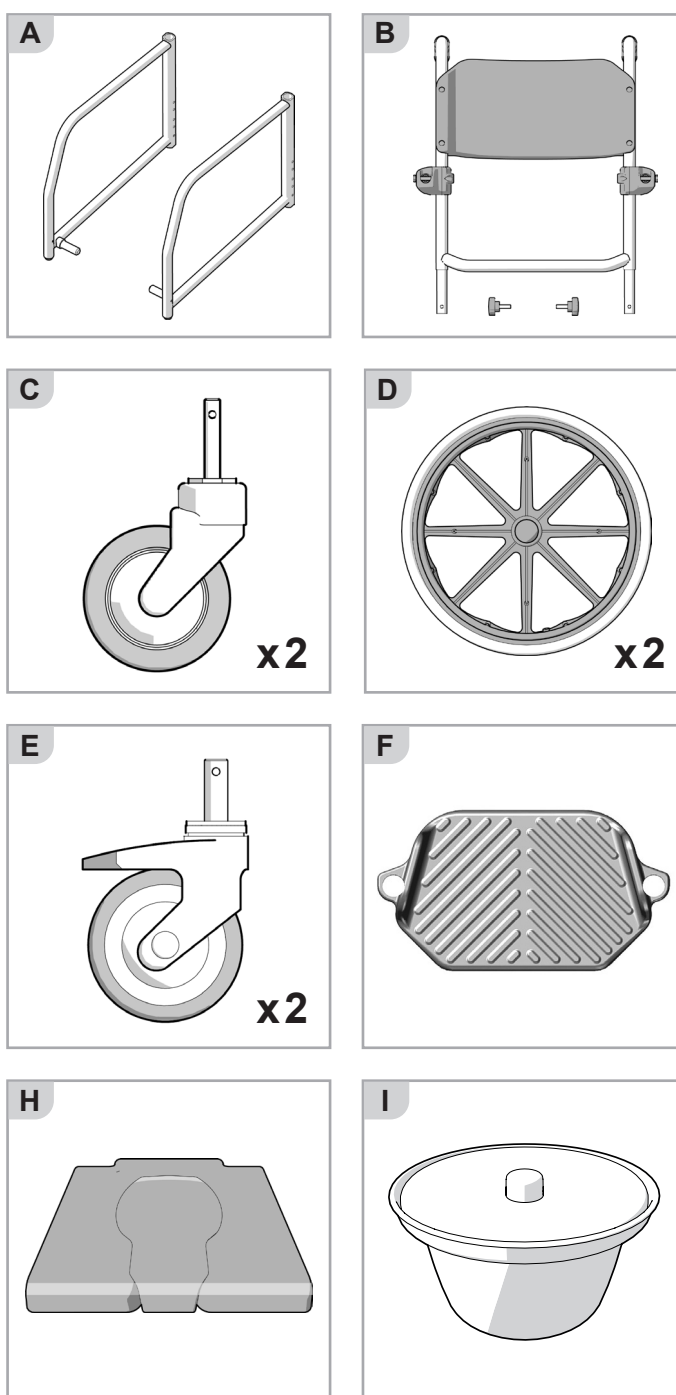
The delivery contains:

- (A) Frame (left & right)
- (B) Backrest with 2 Starknobs
- (C) 2 Front castor wheels (with fixation material)
- (D) 2 Rear wheels or (E) 2 Rear castor wheels
- (F) Footrest
- (G) Connecting bar with 2 Starknobs
- (H) Seat and seat insert
- (I) Toilet bucket

NOTE: For assembly instructions please see Section on pages 6-7 of the manual.

Attention:

The assembly should always be done by an experienced person / authorized dealer

9.0 Handling

Getting into your wheelchair on your own

- Push the wheelchair to a wall or a solid piece of furniture;
- Apply the brakes;
- The user can sit down into the wheelchair;
- Flip down the footboard and position the feet on it (Fig. 9.1).

Getting out of your wheelchair on your own

- Apply the brakes
- Flip up the footboard
- With one hand on the wheel or armrest, the person should lean forwards slightly, to transfer the body weight to the front edge of the seat and then push up to an upright position with both feet firmly on the floor and one foot behind the other (Fig. 9.2).

Fig. 9.1


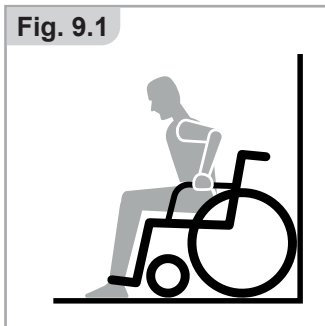
Fig. 9.2

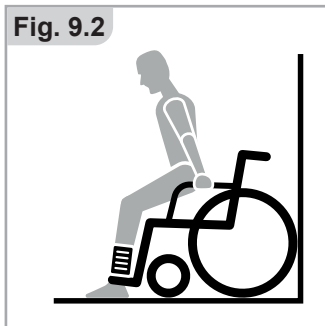⚠ WARNING!

Risk of falling - Never use the seat insert for support when transferring in / out of the wheelchair.

10.0 Brakes

Wheel Locks

Your wheelchair is equipped with two wheel locks. They are applied directly against the tyres. To engage, press both brake levers forward against the stops. To release the locks, pull the levers back to their original positions.

⚠ WARNING!

Braking power will decrease with:

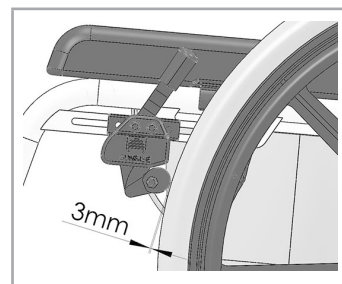
- Incorrect fitting and adjustment of the brakes.
- Worn tyre tread
- Wet tyres
- Improperly adjusted wheel locks
- Use on slippery surfaces

Wheel locks have not been designed to be used as brakes for a moving wheelchair.

The wheel locks should therefore never be used to brake a moving wheelchair.

Always use the hand-rims for braking.

Make sure that the gap between the tyres and wheel locks complies with given specifications. To re-adjust, loosen the screw and set the appropriate gap. Then re-tighten the screw.



11.0 Footboard

The Delphin wheelchair can be shortened considerably by folding the footboard into the frame. In addition it makes it easier to transfer to/from the chair
-> To fold in the footboard, grasp the tube and pull it upward (Fig. 11.1).

Lower leg lengths

By undoing the clamp screw (1) (Fig. 11.2), you can adjust the lower leg length. Loosen the clamp screw and adjust the footboard to the desired position.

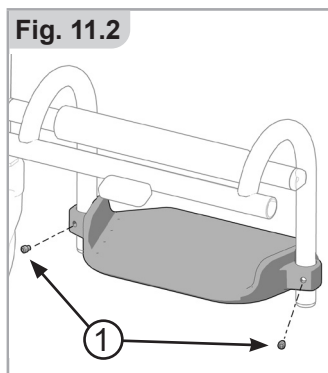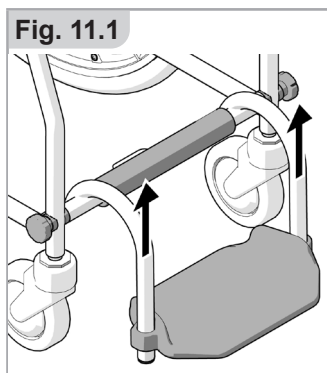Tighten clamp screws again (3 Nm).

! WARNING!

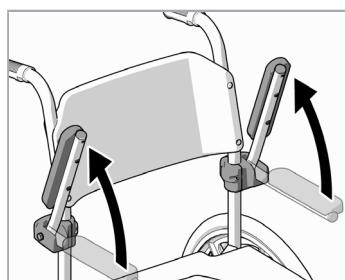
Do not stand on the footplate during transfer. The chair is likely to tip to the front when standing on the footplate. Always place your feet securely on the ground for transferring into or out of the chair.

The footrests are not designed to be used to lift or carry the wheelchairs with an occupant.

A minimum interval of 25mm from the ground should always be maintained.

12.0 Armrests

The arm rest can be flipped up to the rear if you require more freedom of movement while washing or if you wish to change over wheelchairs.


! WARNING!

Finger trap – do not touch the armrest receiver when operating the armrest up or down.

13.0 Seat

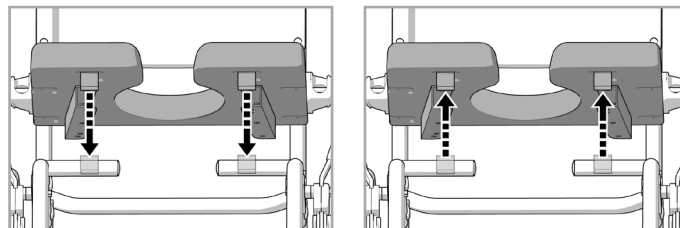The seat can be removed e.g. for cleaning. To do this, grab the seat in the front and the rear and pull it upward. The seat can be attached by pressing it on the frame until all 6 clips snap in.

Attention!

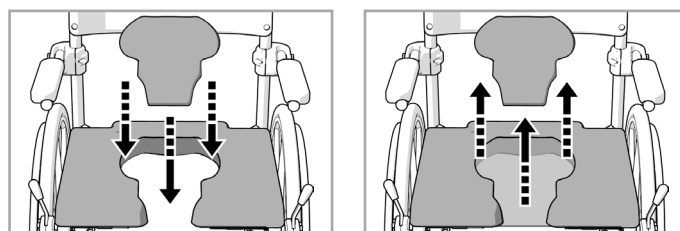
Always make sure that the seat is attached correctly to the frame and all clips are snapped in correctly.

! WARNING

Finger trap – do not grab under the seat when pressing it down

Removing the insert cushion


Before using the toilet the insert cushion must be removed. To do this, pull the insert cushion upwards. To bring it back into position just place it back into the opening.

14.0 Toilet bucket

The DELPHIN can accept a toilet bucket when used as a toilet wheelchair.

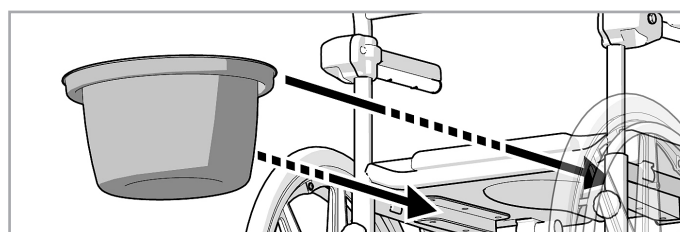- Slide the plastic bucket from the rear into the U-rails provided for this purpose under the seat.
- Make sure that the bucket is correctly locked into the U-rails.

Attention:

After each use the toilet bucket and the seat must be cleaned and disinfected.

15.0 Rear wheels

24" Drive wheels

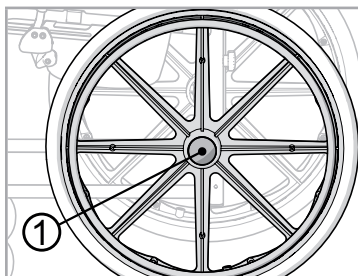
The 24" drive wheels are equipped with quick release axles

Quick-Release Axles for Rear Wheels

Removing the rear wheels will keep the wheelchair as compact as possible.

The wheels can thus be installed or removed without using tools.

To remove a wheel, simply press the quick-release button on the axle (1) and pull the wheel out.


⚠ CAUTION!

Hold the quick-release button on the axle pressed when inserting the axle into the frame to mount the rear wheels. Release the button to lock the wheel in place. The quick-release button should snap back to its original position. Never remove a rear wheel when a person is sitting in the chair.

Lockable Castors

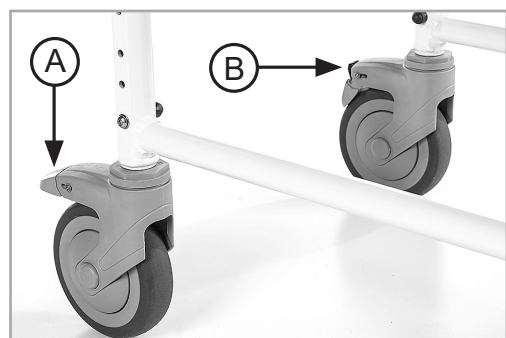The lockable castors are steerable and breakable castors which assume the function of the rear wheels when removed. On the one hand, they make it possible to push the wheelchair through narrow door openings (e.g. to bathroom or WC), on the other hand, they make it easier for the person to maneuver the wheelchair sideways.

Locking:

Press the red lever (A) down with your foot as far as possible to lock the castor

Unlocking:

Press the black release lever (B) with your foot forward to unlock the castor.

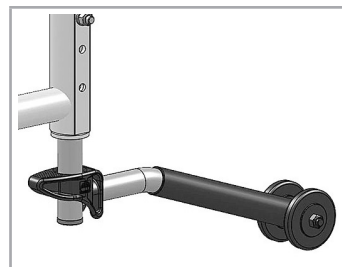
Attention:

When fixing the castors, always make sure that the lockable castors are turned to the rear in order to provide sufficient stability against overturning.

16.0 Anti-Tip Tubes

The anti-tip tubes provide additional safety for inexperienced users when they are still learning how to operate their wheelchairs. Anti-tip tubes prevent a wheelchair from tipping over backwards. The anti-tip tube can be swung downwards by pushing it or rotated forwards. There should be a gap of 30 to 50mm to the floor.

To go up or down a step (e.g. kerb) the anti-tip tubes must be swung forwards, to prevent them from touching the ground.

⚠ WARNING!

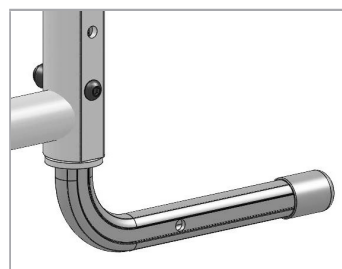
Incorrect setup of the anti-tips will increase the risk of a rearwards tip. The anti-tip wheel protects the wheelchair from overturning backwards.

Note The anti-tip wheel is particularly useful for inexperienced wheelchair users as it gives them more security.

17.0 Step Tubes

Step tubes are used by attendants to push a wheelchair over an obstacle. Simply step on the tube to push a wheelchair, for example, over a step.


Slide the step tube into the rear frame tube and fix it with the M6 screw and a self-locking nut.

18.0 Safety bar

The safety bar can be flipped up for getting in and out of the wheelchair.

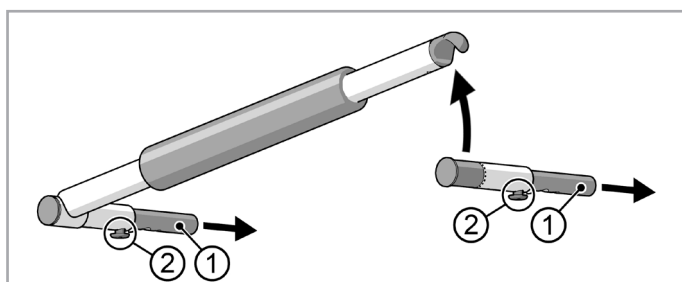To assembly the safety bar the end plugs of the armrest tube must be removed first. Then slide the insert tubes (1) into the armrest tube and lock them.

To remove the retaining bar press (2) and pull it out the front.

! WARNING!

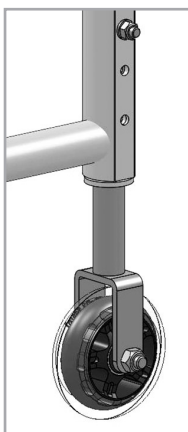
Always make sure that the safety bar locks in correctly to avoid injuries.

19.0 Transit wheels

Travel wheels should be used wherever your wheelchair would be too wide if the rear wheels were used. After the rear wheels have been removed with the aid of the quick-release axles, the transit wheels can immediately be used to continue riding.

The transit wheels are mounted so that they are approx. 30 mm above the ground when not in use. They are thus out of the way when riding, transporting, or when tipping to pass over obstacles.

Slide the transit wheel into the back tube and fix it with the M6 screw and a self-locking nut.


! CAUTION!

Without rear wheels, your wheelchair has no wheel locks.

20.0 Crutch holder

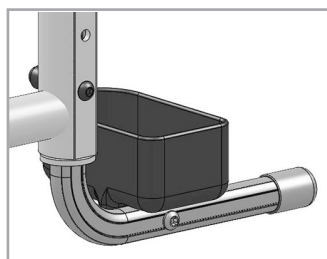This device permits crutches to be transported directly on the wheelchair. It has a Velcro loop to fasten crutches or other aids.

! CAUTION!

Never try to use / remove the crutches or other aids while moving.

21.0 Name Plate

The nameplate is located on either the cross-tube assembly or the transverse frame tube, as well as on a label in the owner's manual. The nameplate indicates the exact model designation and other technical specifications. Please provide the following pieces of information whenever you have to order replacement parts or to file a claim:

- Serial number
- Order number
- Month/Year.

		Sunrise Medical GmbH Kahlbachring 2-4 D-69254 Maisch / Germany		2019-11-19
		TYPE: Toilettensstuhl SN 203194721748065 Delphin ISO 7176-19:2008		
90 kg	max 10°	430 mm	420 mm	

UK	RP	Sunrise Medical Limited Thorns Road, Brierley Hill West Midlands, DY5 2LD UNITED KINGDOM
-----------	-----------	---

TYPE:	Product Name/SKU Number.
max X°	Maximum safe slope with anti-tip tubes fitted, Depends on wheelchair setting, posture and physical capabilities of the user.
XXX mm	Seat width.
XXX mm	Seat depth.
XXX kg	Load Maximum.
	UKCA Mark.
	CE Mark.
	Consult instructions for use.
XXXX-XX-XX	Date of manufacture.
SN	Serial number.
MD	This symbol means Medical Device.
	Manufacturer's address.
ISO 7176-19:2008	Crash tested according to ISO 7176-19:2008.
	Importer's address
UK RP	UK Responsible Person
CH REP	Swiss Representative's address

22.0 Maintenance and care

If your wheelchair should ever get wet, please dry it afterwards.

A small amount of sewing-machine oil should be applied to quick-release axles approximately every 8 weeks.

Depending on the frequency and type of use, we recommend taking your wheelchair to your authorized dealer every 6 months to have it inspected by trained personnel.

If you want to store the wheelchair for a long period of time, then no further measures are required. Make sure that the wheelchair is stored at room temperature in a dry place which is protected from strong sunlight. Before using it again, the wheelchair should be checked by an authorized dealer.

When	What	Comment
Before use	Check brake function	
	Check all components for any damages	If damages are visible do not use the chair and contact your dealer
	Check if the seat is correctly fitted	
	Check if the back is correctly fitted	
	Check that all starknob screws are tightened	
	Check correct locking of the Quick release axle	
Every 8 weeks (depending on use)	Lubricate the Quick release axles with a few drops of oil	
	Check all screw connections for tight fit	
After use in salty or chlorine water	Wash chair with regular tap water and dry it afterwards	

CAUTION!

Sea / Salt / chlorinated water can damage the bearings of the front and rear wheels. Clean the wheelchair thoroughly after exposure e.g. by regular tap water.

Cleaning and disinfection:

Silicone-free, water-based cleaning and care products should be used for cleaning and care.

Please follow the manufacturer's instructions for use. Clean the upholstery with warm water and hand dishwashing liquid. Stains can be removed with a sponge or a soft brush. Wipe off stubborn dirt using a commercially available mild detergent. Wipe with clean water and allow drying.

For disinfection, spray the surfaces with a disinfectant after cleaning. For this purpose, a disinfectant from the DGHM list must be used, e.g. Antifect Liquid (Schülke & Mayr) for rapid alcohol-based disinfection for medical devices and equipment that need to be disinfected quickly.

Please follow the manufacturer's instructions for use for the disinfectant you are using.

CAUTION!

- Do not spray the shower and toilet wheelchair with a high-pressure cleaner!
- Only use cleaning agents with a pH value of at least 6 (neutral cleaners).
- Do not use aggressive cleaning agents, e.g. solvents, or hard brushes, etc.

Hygiene measures when being re-used:

Prior to the wheelchair being re-used, it must be carefully prepared. All surfaces which come into contact with the user must be treated with a disinfection spray.

To do this, you must use a disinfectant as authorised/ recommended in your country, for rapid alcohol-based disinfection for medical products and medical devices, which must be disinfected quickly.

Please be aware of the manufacturer's instructions for the disinfectant you are using.

In general, a complete disinfection cannot be guaranteed on seams. We therefore recommend that you dispose of seat and back slings to avoid micro-bacterial contamination with active agents according to your local infection protection law.

Storage

To avoid the risk of damaging the product:

- Store the product in a dry area.
- Do not store the product near heat sources.
- Do not expose the product to direct sunlight.

Please also see storage and shipping conditions chapter 25.

23.0 Trouble shooting**Wheelchair pulls to one side**

- Check to make sure wheel turns easily (bearings, axle)
- Check to make sure both castors are making proper contact with the ground

Castors begin to wobble

- Check to make sure all bolts are secure; tighten if necessary (see the section on torque)
- Check to make sure both castors are making proper contact with the ground

Wheelchair squeaks and rattles

- Check to make sure all bolts are secure; tighten if necessary (see the section on torque)
- Apply small amount of lubrication to spots where movable parts come in contact with one another

Wheelchair begins to wobble

- Check if rear wheels are adjusted correctly
- Check if all starknob screws are tightened correctly
- Check if seat and back are placed correctly

Parking brake of the self-propelling wheels does not engage

- Parking brake may be wrongly adjusted -> The parking brake must be adjusted by a qualified technician.

The castors are difficult or impossible to turn

- There may be dirt between the fork and the rim
->Thoroughly clean the castors

Difficulty moving the footrests up

- Deposits may be in the guides -> Thoroughly clean the footrests and the guides

24.0 Disposal / recycling of materials

NOTE: If the wheelchair has been supplied to you free of charge it may not belong to you. If it is no longer required follow any instructions given by the organization issuing the wheelchair in order that it may be returned to them.

The following information describes the materials used in the wheelchair in relation to the disposal or re-cycling of the wheelchair and its packaging.

Specific waste disposal or recycling regulations may be in force locally and these should be taken into consideration when disposal arrangements are made. (This may include the cleaning or de-contamination of the wheelchair before disposal).

Aluminum:	side frame, armrests, leg rest, backrest
Stainless Steel:	Fasteners, QR axle, axle adapter
Composite:	Handgrips, tube plugs, castor wheels, footplate, armpads, seat, back, armrest receiver, wheels, tires
Packaging:	Cardboard box

Disposal or recycling should be done through a licensed agent or authorized place of disposal. Alternatively your wheelchair may be returned to your dealer for disposal.

25.0 Technical Specifications

Technical Specifications

Overall width:

With standard wheels including handrims: SW + 190 mm

Weight in kg:

- Transportation (without wheels & seat) 9.9 kg
- Wheels (unit) 2.7 kg

Maximum user weight:

- Delphin up to a load of 130 kg

ADDITIONAL NOTES

In Accordance with EN12183: 2009 all there listed parts (seat - back upholstery, arm pad, side panel...) are fire resistant as per EN1021-2.

WARNING!

The Dynamic safe slope is dependent on the chair configuration, the user's abilities and the style of riding. As the users abilities and style of riding cannot be predetermined then the max safe slope cannot be determined. Therefore this must be determined by the user with the assistance of an attendant to prevent tipping.

It is strongly recommended that inexperienced users have Ant-tips fitted.

The wheelchair conforms to the following standards:

- Requirements and test methods for static, impact and fatigue strengths (ISO 7176-8)
- Requirements for resistance to ignition in accordance with ISO 7176-16 (EN 1021-1)

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical SAS