JAY®

J2 & J2 DC CUSHION

J2 & J2 DC Cushion

Important Consumer Information

NOTE: This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read this entire manual and save for future reference.

WARRANTY REGISTRATION: To validate the warranty on this product, please complete and return the postcard attached to the enclosed instruction and warranty booklet.

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NOTE:

Check all parts are free from shipping damage. In case of damage DO NOT use. Contact Sunrise Medical or your supplier for further instructions.

Sunrise Medical recommends that a clinician such as a doctor, therapist or rehab engineer experienced in seating and positioning be consulted to determine if a J2 & J2 DC Cushion is appropriate.

Cushions should only be installed by:

- An authorized Sunrise Medical supplier
- A qualified healthcare professional who has undergone appropriate training by either Sunrise Medical or their approved supplier.

GENERAL WARNING

- Before using this cushion, you and each person who may assist you, should read this entire Manual and make sure to follow all instructions. Review the Warnings often, until they are second nature to you.
- DO NOT install this equipment without first reading and understanding this manual. If you are unable to understand the Warnings, Cautions and Instructions, contact a qualified clinician or supplier - otherwise injury or damage may occur.
- Hook and loop Velcro® should not come into prolonged contact with moisture as this may degrade the adhesive and lead to a failure of the Velcro®.

INTRODUCTION

The J2 and the J2 DC cushions have been engineered to provide a high degree of pressure reduction and postural support with a minimum of maintenance.

To ensure proper use of the cushion, please read these instructions and save them for future reference.

The intended lifetime of this product is 5 years.

Maximum User Weight:

- **J2 = 175 kg**
- **J2 DC = 150 kg**

Word	Definition
 DANGER!	Advice to the user of potential risk of serious injury or death if the advice is not followed
 WARNING!	Advice to the user of a risk of injury if the advice is not followed
 CAUTION!	Advice to user that potential damage to equipment may occur if the advice is not followed
NOTE:	General advice or best practice
	Reference To Additional Documentation

WARNING

- Installing a cushion on a wheelchair may affect the centre of gravity of the wheelchair and may cause the wheelchair to tip backwards, potentially resulting in injury. Always assess the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.
- Prior to prolonged sitting, any cushion should be tried for a few hours at a time while a clinician inspects your skin to ensure that red pressure spots are not developing. You should regularly check for skin redness. The clinical indicator for tissue breakdown is skin redness. If your skin develops redness, discontinue the use of the cushion immediately and see your doctor or therapist.
- The J2 & J2 DC cushion is designed to help reduce the incidence of pressure ulcers. However, no cushion can completely eliminate sitting pressure or prevent pressure ulcers.

WARNING

- Do not leave the cushion outside overnight at temperatures below 40° F (5° C). Allow the cushion to warm to room temperature before using. Sitting on extremely cold or hot surfaces can cause skin damage.
- Avoid sharp objects or exposure to freezing, direct sunlight, excessive heat or open flame.
- **OBSTRUCTIONS: DO NOT:** place any obstructions between the user and the cushion because it will reduce product effectiveness.
- **CUSHION AND COVER ORIENTATION:** Product must be used the correct way up. If the cover is not used correctly or if the cover is the wrong size, it may reduce or eliminate the cushion's benefits and could increase risk to the skin and soft tissue.

FITTING THE J2 CUSHION (WITHOUT USER SEATED)

Place the cushion in the chair with the material identification tags facing the rear of the chair. The product label should be visible on the right side of the cushion.

NOTE:

A solid base of support, like the Jay Adjustable Solid Seat or Solid Drop Seat Insert is recommended.

If using an adjustable depth back on the wheelchair, like the JAY 3 Back, position the back of the Cushion inline with the front of the upright back posts.

If using a fixed depth back, like a JAY®J2 Back or Jay Care™ Back, notch the rear corners of the base to establish seat depth to within 25 - 50 mm, (1" - 2"), from the back of the knees (see score lines on base). This will allow the back of the Cushion to go between the posts.

FITTING THE J2 CUSHION (WITH USER SEATED)

Place the individual on the Cushion and adjust the leg rests to allow the buttocks to sit in the rear, carved-out portion and the legs to rest comfortably on the cushion.

To achieve the lowest average sitting pressures possible, the footrests should be adjusted low enough to allow the legs to rest firmly on the front of the Cushion.

The Cushion should come within 25 - 50 mm, (1" - 2"), from the back of the knees.

Assess the client.

- Are the hips level?
- Are the legs in good position?
- Is the client comfortable?

If the answer to any of these questions is "no," consult your healthcare professional.

NOTE:

- Proper footrest adjustment will enhance sitting comfort.
- Do not use the wheelchair's seat width as a guide in determining the appropriate cushion width. Improper cushion fit may be determined to the performance of a pressure distributing cushion and its support of the body.

JAY® J2 AND J2 DC CUSHION

The J2 Cushion has been engineered to provide a high degree of pressure reduction and postural support with a minimum of maintenance.

The J2 Deep Contour Cushion is the newest member of the J2 cushion family. The J2 Deep Contour Cushion was developed for clinicians to use with clients who are at extremely high risk for skin breakdown.

Designed for superior skin protection, pressure management and stability, the J2 Deep Contour Cushion is a maintenance free cushion designed for ease of use.

J2 Cushion Components

Fig.1.0, JAY Flow Fluid Pad

Fig.2.0, Shaped Foam Base with Velcro strips.

Fig.1.0

Fig.2.0

FITTING THE J2 DEEP CONTOUR (DC) CUSHION

! WARNING:

- Please read the "WARNINGS" Section at the beginning of this document.

Select the cushion to match your hip width when in the seated position. The cushion length should extend to within 25 mm - 50 mm, (1"-2"), from the back of the knees. Place the cushion to the back of the wheelchair with the material identification tags and fluid pad toward the rear. When fit correctly, your hips should be to the back of the chair. Your ischials (seat bones) should be centered on the Jay Flow™ Fluid pad, (Fig.4.0), in the seat well of the cushion, (Fig.5.0).

Proper footrest adjustment may enhance sitting comfort and help lower peak sitting pressures. Adjust the footrests so your legs rest firmly but comfortably on the cushion.

NOTE: Do not use the wheelchair's seat width as a guide in determining the appropriate cushion width. Improper cushion fit may be detrimental to the performance of a pressure-distributing cushion and its support of the body.

! CAUTION:

- Avoid using the cushion on overly stretched seat upholstery. Stretched upholstery does not provide adequate support for the cushion and improper positioning may result.
- Replace stretched upholstery or purchase the J2 Deep Contour Solid Seat Insert or order the J2 Deep Contour Cushion with the Solid Drop Seat. The Solid Drop Seat Option is attached to the cushion for a one piece design that replaces the wheelchair's seat upholstery, (Fig.3.0).
- Avoid using overly stretched back rest upholstery. Stretched back upholstery does not provide adequate support for the body and improper positioning may result.
- Replace stretched back rest upholstery or purchase a back support product from Sunrise that is appropriate for you. Contact an authorised supplier for a free demonstration of these products.

J2 DC CUSHION COMPONENTS

Fig.3.0

Fig.4.0

Fig.5.0

CHECKING FOR BOTTOMING OUT, J2 J2 DC.

Bottoming out occurs on a J2 J2 DC cushion when you displace the fluid out from underneath your seat bones, which leaves you sitting on the foam base. It sometimes occurs on very thin individuals, people using recliner wheelchairs, people who slouch when sitting or if using a cushion that is too wide. When bottoming out occurs, increased pressure is loaded onto the ischials and coccyx increasing the risk for skin breakdown.

To check for bottoming out, sit on the cushion without the cover for a minimum of two minutes. Transfer up and off the cushion (or have someone help you transfer), trying not to disturb the fluid underneath you. Push down in the depressions on the pad where your ischials (seat bones) and coccyx (tailbone) were. You should have to push through at least 10 mm (1/2") of fluid before you feel the firm cushion base below.

If the cushion is properly positioned and the footrests are properly adjusted, and there is not at least the minimum 10 mm (1/2") of fluid, the cushion is bottoming out and should not be used.

If you are bottoming out, discontinue use of the cushion and see your clinician. Usually bottoming out is easily solved by using fluid supplement pads (part# F119). Call your local authorised supplier to see if this appropriate for you.

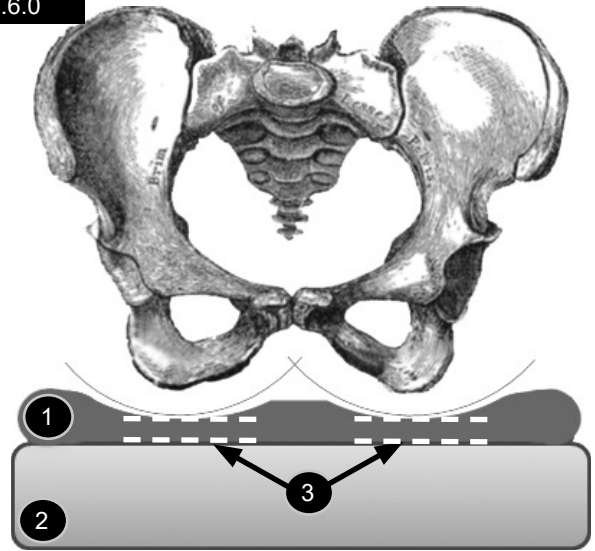
NOTE: We recommend using a JAY® Adjustable Solid Seat in place of your wheelchair upholstery or a J2 Solid Seat Insert in addition to your upholstery. It will optimize the performance of your J2 J2 DC Cushion and help provide better positioning for you.

⚠ WARNING:

Failure to check the Pressure Relief Pad for bottoming out can result in skin redness or skin breakdown.

Maximum User Weight:

- J2 = 175 kg
- J2 DC = 150 kg

Fig.6.0

- 1 Fluid Pad.
- 2 Foam Base.
- 3 Minimum 10.0 mm Fluid Depth.

EASY MAINTENANCE AND CLEANING

Regular cleaning and maintenance may help extend the life of your cushion. During cleaning, component inspection is recommended.

Check cover for tears and excessive wear:

1. Remove cover and check for abnormalities (inside and out).
2. Check the foam base for breakdown or degradation.

Cleaning the cover:

1. Remove the outer cover from the foam base and zip closed.
2. Machine wash in warm water and drip or tumble dry on low heat. No ironing is required or suggested.

Sterilisation Advice:

CAUTION:

Sunrise Medical does not recommend high temperature sterilization procedures.

If local protocol requires sterilization, please consult the instruction table at the end of this manual. The cushion cover may be washed at 80°C for 10 minutes and may be tumble dried, but we recommend cleaning the cover by using the shortest time and the lowest temperature allowed by local protocol used.

To refit the cover after washing, fit cover back over foam base. Ensure the back of the cover matches up to the back of the foam base.

NOTE: If utilised in an institutional setting, write the resident's name with a permanent marker on the content label of the cover. This will ensure the return of the correct cover to its matching cushion after washing.

Cleaning the foam base:

1. Remove the outer cover from the foam base and wipe lightly with a damp cloth. Do not use soap. Do not submerge in water. Wipe off with a clean cloth.
2. Let your cushion completely air-dry before reassembly.
3. Cleaning the fluid pad(s):
 - d. Remove the cover from the foam base.
 - e. Remove fluid pad(s) and wipe with warm water and soap.
 - f. Wipe off with a clean cloth.
 - g. Reinstall fluid pad(s), then fit cover back over foam base.

NOTE: Ensure the back of the cover matches up to the back of the foam base.

4. Cleaning accessory parts:

- d. Remove the cover from the foam base and fluid pad(s), if necessary, to clean accessory parts.
- d. Remove accessory parts. Wipe with warm water and soap.
- d. Wipe off with a clean cloth.
- d. Reinstall accessory parts and fluid pad(s), then fit cover back over foam base.

NOTE: Ensure the back of the cover matches up to the back of the foam base.

REASSEMBLING THE J2 / J2 DC CUSHION AFTER CLEANING

Follow cleaning and maintenance instructions as described in "Easy maintenance and cleaning" then reassemble as follows:

1. Attach the fluid pad to the foam base.
2. Attach the rear three Velcro® strips first, then the front two Velcro® strips.
3. The pleats in the rear of the pad will automatically wrinkle the pad for you.
4. There should be plenty of wrinkles in the fluid pad.
5. This maximizes pressure reduction and helps reduce shearing by allowing the pad to conform to bony prominences.

HYGIENE MEASURES WHEN BEING RE-USED

Prior to the wheelchair being re-used, it must be carefully prepared. All surfaces which come into contact with the user must be treated with a disinfection spray.

To do this, you must use a disinfectant as authorised/ recommended in your country, for rapid alcohol-based disinfection for medical products and medical devices, which must be disinfected quickly.

Please be aware of the manufacturer's instructions for the disinfectant you are using.

In general, a complete disinfection cannot be guaranteed on seams. We therefore recommend that you dispose of seat and back slings to avoid micro-bacterial contamination with active agents according to your local infection protection law.

CONDITIONS FOR SAFE STORAGE

- Products should be stored in a dry place: at 20 – 75% Relative Humidity.
- Protect from direct sunlight and dust for long periods.
- Away from direct heat and at a temperature between 5°C (40°F) to 40°C (104°F)
- Position and stack the packaged products according to the "this side up" arrows on the packaging.
- If a package is high and narrow and therefore potentially unstable, ensure it is well secured on the pallet or in the racking, to prevent it from falling down.
- Do not stack anything on top of the packaged products when the "do not stack" mark on the packaging is shown.
- Product should never be stored outside and subject to the elements.

FITTING A PRESSURE DISTRIBUTING CUSHION

1. Select a cushion to match your hip width when in the seated position.
2. The cushion length should extend to within 25 mm - 50 mm (1" - 2") from the back of the knees.
3. Place the cushion to the back of the wheelchair with the material identification tags and fluid pad(s) toward the rear.
4. When fitted correctly, your hips should be to the back of the chair and your ischials, (seat bones), should be centered on the JAY Flow™ Fluid pad(s) in the seat well of the cushion.
5. Proper footrest adjustment may enhance sitting comfort and help lower peak sitting pressures.
6. Adjust the footrests so your legs rest firmly but comfortably on the cushion.

⚠ CAUTION:

- Do not use the wheelchair's seat width as a guide in determining the appropriate cushion width.
- Improper cushion fit may be detrimental to the performance of a pressure distributing cushion and its support of the body.
- Avoid using the cushion on overly stretched seat upholstery. Stretched upholstery does not provide adequate support for the cushion and improper positioning may result.
- Replace stretched upholstery or purchase the JAY Adjustable Solid Seat or the JAY Solid Seat insert that is appropriate for the cushion.
- Avoid using overly stretched back rest upholstery. Stretched back upholstery does not provide adequate support for the body and improper positioning may result. Replace stretched back rest upholstery or purchase a back support product from Sunrise that is appropriate for you. Contact an authorised supplier for a free demonstration of these products.

ASSEMBLING AND ADJUSTING THE SOLID DROP SEAT

Located on the bottom of the Solid Drop Seat (1) are 10 pairs of holes (3), five pairs each side of the platform. Each hook has two slots that line up with one pair of holes, (Fig.7.0 - 8.0)

1. Locate where the hooks (2) should best be placed on the platform (1) so that the installation doesn't interfere with hardware on the wheelchair seat frame.
2. There are four hooks (2).
3. Each side requires installing two hooks.
4. Align the slots in the hooks (2) with one pair of holes (3).
5. Slide a washer onto a bolt and fasten the bolt through the slot in the hook into the Solid Drop Seat.
6. Each hook requires two bolts.
7. Do the same for all four hooks.
8. Leave all hooks slightly loose.
9. Place the solid seat on the seat rail frame of the wheelchair and adjust the width.
10. Tighten as many hook bolts as possible prior to removing Solid Drop Seat from the wheelchair, (this will preserve their alignment).
11. Remove the Solid Drop Seat from the wheelchair and finish tightening the bolts to secure the hooks.

Fig.7.0

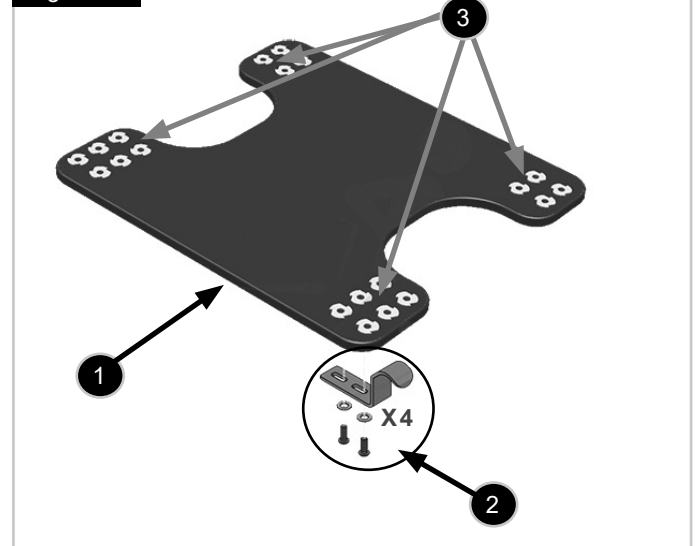
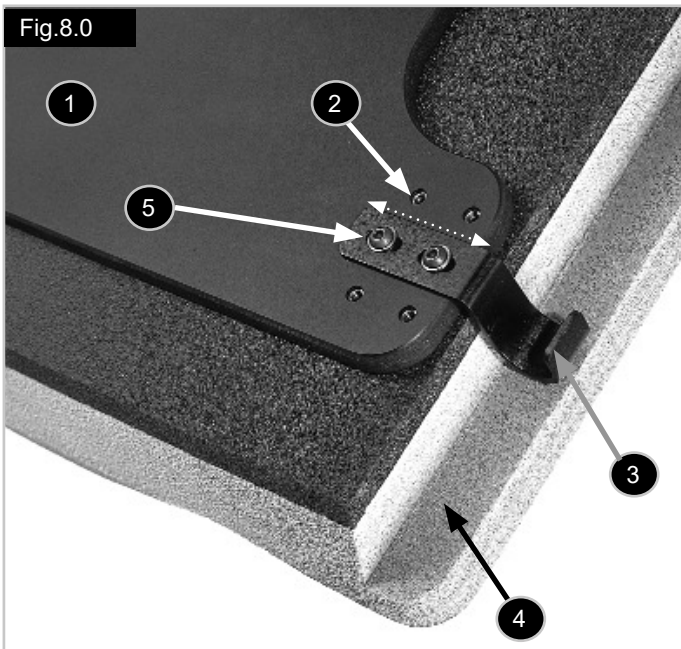
Fig.8.0

**Figure 8.0 labels:**

1. A strong platform base promotes level sitting surface.
2. Five, hook attachment locations accommodate most wheelchair frames.
3. Drop Hooks can be used to change the seat to floor height. Available in; 12.7 mm, 25.4 mm, 38.1 mm, (1/2", 1", 1-1/2"), sizes.
4. The cushion is notched to allow for a drop of up to 38.1 mm, (1-1/2").
5. Each hook can be slide adjusted up to 12.7 mm, (1/2"), to allow for slight differences in wheelchair widths.

SECURE HOOK RETAINERS AND STOPPER KIT

To secure the restraining hardware onto your wheelchair rails:

6. Snap the two hook retainers toward the back of the wheelchair seat rail.
7. The rear hooks will slide under these as shown below. These will prevent your cushion from accidentally popping out of the wheelchair.
8. Snap the hook stopper towards the front of the wheelchair seat rails.
9. The front hooks rest behind the stopper to prevent your cushion from sliding forward.
10. Retainers and stoppers should be secured tightly to wheelchair seat rails with the nuts and bolts provided.
11. Point the heads of the bolts to the outside of the wheelchair.

⚠ CAUTION:

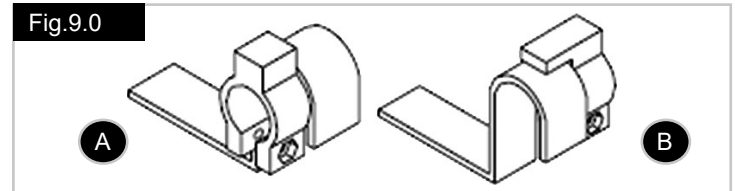
Enclosed are three lengths of 8-32" bolts. Using a bolt longer than the required length can interfere with installation and removal of seat.

Use the following sizes contained in the kit:

- 8-32 x 3/4" when the stopper and/or retainers are mounted on 7/8" tubing.
- 8-32 x 1" when the stopper and/or retainers are mounted on 1" diameter tubing.
- 8-32 x 1-1/4" when the stopper and/or retainers are mounted over "rivnut" threaded insert on 1" diameter tubing.
- To install cushion, simply slide rear hooks under the rear hooks retainers and drop front hook behind hook stopper.

Note: If hook retainers or stopper cannot be installed due to interference with hardware on your wheelchair, you might need to select an alternate location for your brackets and hooks.

Fig.9.0

**Figure 9.0 labels:**

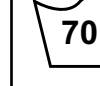
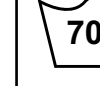
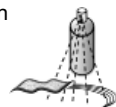
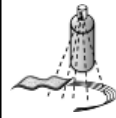
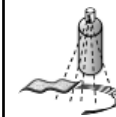
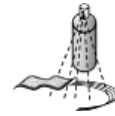
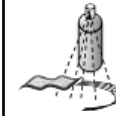
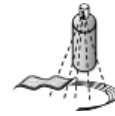
- A = Use hook stoppers 1/4" forward of the front hooks.**
B = Use hook retainers behind rear hooks.

Length adjustment:

For a longer seat length, position Solid Drop Seat 25.4 mm (1") forward of the rear vertical posts of your wheelchair. If front hooks are less than 25.4 mm (1") from front end of seat rails, then remove front hooks and move them to the alternate location directly behind (see picture).

Fitting Hints:

Once you have determined the position of your Solid Drop Seat on your wheelchair, you may find that the hooks interfere with the hardware that is particular to your wheelchair. If this is the case, reposition those particular hardware hooks to an alternate location.

Approved Cleaning Procedures for JAY Cushions									
Product	JAY Lite	JAY J3	J2 GS	JAY Easy Visco	JAY Soft Combi P	JAY Xtreme Active	JAY Basic	JAY Easy	JAY Balance
Outer Cover (Microclimatic and Stretch)									
(1). Wet Wash 70°C 									
Outer Cover (Incontinent)									
(2). Wet Wash 60°C 									
Inner Cover									
(3). Spray and Wipe disinfection  or Wet Wash 60°C	NA	NA	NA	NA	NA		NA	NA	
Cushion core (foam base)									
(4). Steam autoclave up to 105 °C (1) 									
Spray and wipe disinfection 									
(5). Immersion disinfection 									
Cushion elements									
Air element	NA		NA	NA	NA	NA	NA	NA	
Positioning elements	NA			NA	NA		NA	NA	
Fluid pad	NA			NA	NA		NA		
1. 70°C is the maximum recommended temperature. 2. 60°C is the maximum recommended temperature. 3. The Xtreme Active and Balance inner covers can be washed at 60°C as well as spray and wipe disinfected. Possible discoloration if washed at 60°C but the functional aspect of the covers is not diminished. 4. An autoclave is a self locking machine that sterilizes with steam under pressure. 5. At low temperature (< 30°C)									
The product warranty shall not apply to products damaged due to the failure to follow these cleaning instructions or the use of inappropriate cleaning compounds.									

Recommended Disinfectants

All disinfectants which are listed and recommended by the Robert Koch Institute: http://www.rki.de/EN/Home/homepage_node.html

24 MONTH LIMITED WARRANTY

Each J2 & J2 DC cushion is carefully inspected and tested to provide peak performance. Every J2 & J2 DC cushion is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover.

The removable cover is guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within six (6) months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge.

Claims and repairs should be processed through the nearest authorised supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

ADDENDUM TO WARRANTY USE AND CARE GUIDE

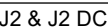
Consult your health care professional for proper fitting and use, and read the following statements carefully:

- The fluid in this cushion may lose volume over time.
- Volume loss in this cushion may result in bottoming-out of the cushion.
- Monthly, during routine cleaning and inspection, check the cushion for bottoming-out (see instructions under "checking for bottoming-out" in the warranty use and care guide) and any fluid inconsistencies. If the fluid is firmer in one area simply knead it back to its original consistency.
- If bottoming-out occurs discontinue use of this cushion and contact your health care professional. If further assistance is necessary, contact Sunrise Medical Customer Services.

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S