Jay Care Back

User Instruction Manual e Warranty

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JAY CARE BACK - 3500 SERIES MODELS

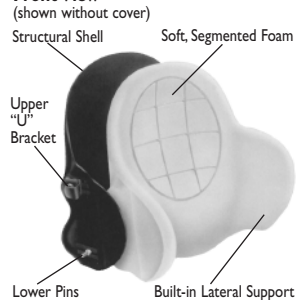
The Jay Care Back is designed to accommodate and support the kyphotic spine and provide a high degree of comfort and stability. It is designed to help increase sitting tolerance and function. Its unique foam construction also helps provide pressure relief along the spine. In addition, it is lightweight, durable and easily adjustable.

Maximum user weight: 250 lbs. (113 kg).


Caution

Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury. Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

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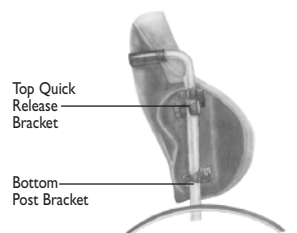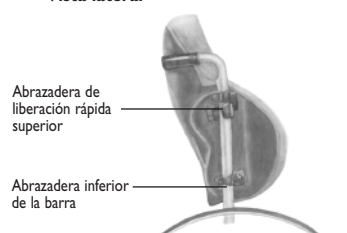
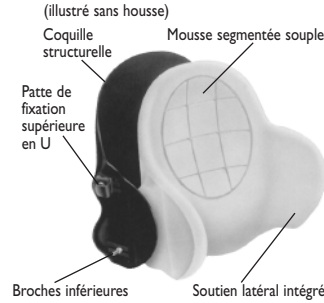
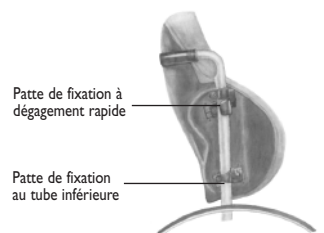
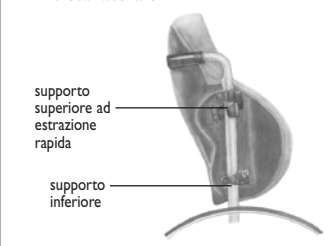
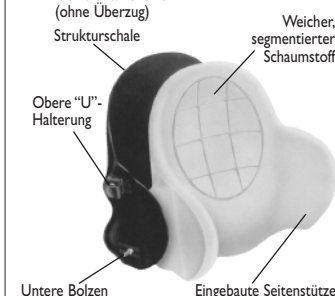
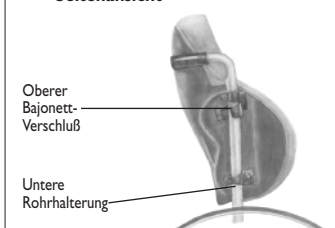
INSTALLING THE JAY CARE BACK


The Jay Care Back is designed to replace the wheelchair's sling upholstery.

The initial installation should be done without the foam insert in the Back shell.

Installing the bottom Universal Post Bracket hardware

1. Remove the upholstery from the back of the wheelchair.
2. Attach one bottom universal post bracket to the bottom of each back post of the wheelchair.(Diagram I)
- a) The universal hardware is designed to fit on 5/8", 3/4", 7/8" and 1" (1.6, 1.9, 2.2 and 2.5 cm) diameter back post tubing. (The bracket comes assembled to attach to 7/8" or 1" (2.2 or 2.5 cm) tubing) Check diagram I to determine which strap hole is appropriate for the installation. Wrap the strap around the wheelchair post. Screw "A" may have to be loosened to align the strap hole with the barrel nut located inside the post bracket body. Place a washer on screw "B" and insert the screw through the appropriate hole in the strap.
- b) Tighten screw "B" into the barrel nut through the side of the post bracket body. Tighten the opposite screw if necessary.
- c) Snap the screw caps over both screw heads.
- d) To adjust the height of the bracket, loosen screw "A" until the bracket slides freely on the back post. Once the proper height of the back is determined, tighten the screw. All future adjustments of the back height are done with screw "A".

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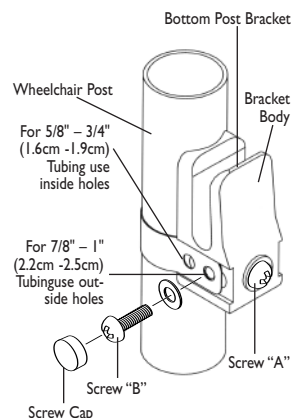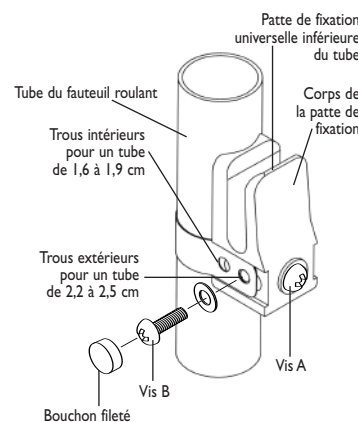
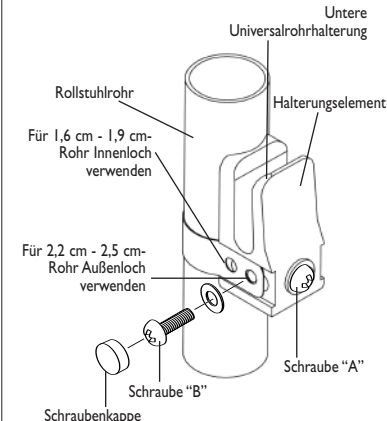
Abbildung I Teilenummer 680C


INSTALLING & ADJUSTING THE JAY CARE BACK

Installing the Jay Care Back

1. Place the Jay Care Back in the wheelchair so that the bottom pins slide into the the bottom Universal Post Bracket hardware receivers. The Back's top hooks should rest against the wheelchair back posts. The bottom pin head should extend past the outside edge of the post bracket receiver.

Note: If the wheelchair measures slightly wider than the usual 16" (40cm), 18" (46cm), or 20" (50cm) widths, it may be necessary to order Extended Hardware, part# M-301. Call your local authorized supplier.

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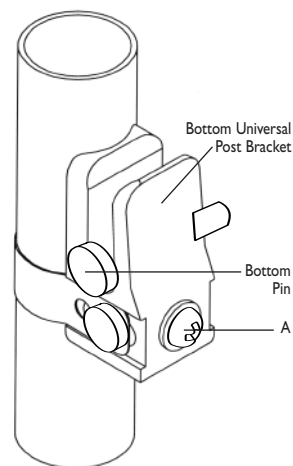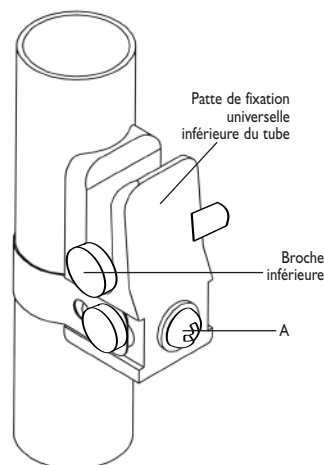
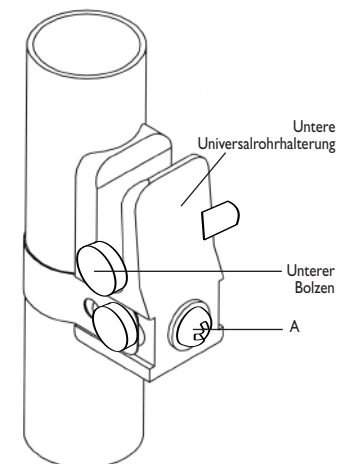
Abbildung 2

ENGLISH

INSTALLING THE TOP QUICK RELEASE QUARTER TURN BRACKET HARDWARE

1. Attach one top quick release quarter turn bracket to each upright post of the wheelchair.
- a) The top quick release quarter turn bracket is designed to fit on 5/8", 3/4", 7/8" and 1" (1.6, 1.9, 2.2 and 2.5 cm) diameter back post tubing. (The bracket comes assembled to attach to 7/8" or 1" (2.2 or 2.5 cm) tubing) Check diagram 3 to determine which strap hole is appropriate for the installation. Wrap the strap around the wheelchair post. Screw "A" may have to be loosened to align the strap hole with the barrel nut located inside the bracket body. Place a washer on screw "B" and insert the screw through the appropriate hole in the strap.
- b) Tighten screw "B" into the barrel nut through the side of the bracket body. Tighten the opposite screw if necessary.
- c) Snap the screw caps over both screw heads.
- d) To adjust the height of the bracket, loosen screw "A" until the bracket slides freely on the back post. Once the proper height of the back is determined, tighten the screw. All future adjustments of the back height are done with screw "A".
- e) Always turn the quick release quarter turn bracket to the "DOWN" position (diagram 4) to secure the top hook in place.

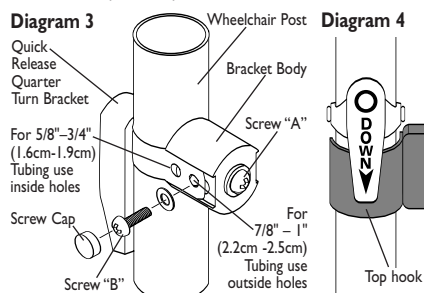
Diagram 4 Note: The back is properly secured when the bottom pins are resting in the lower post brackets and the upper quick release quarter turn brackets are installed above the back support's top hook brackets with the quick release mechanism pointing down.

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Adjusting the back angle

The adjustment of the back angle is accomplished by using the top "slide" bracket screws visible on the sides of the back shell. A 23° recline is possible without losing seat depth. A maximum 39° angle is possible.

1. Use a flat head screwdriver to turn the top screw, securing the upper "hook" bracket on either side of the back shell. Turn the screws one full turn to loosen. The "hook" bracket should slide freely.
2. After loosening the screws, adjust the back forward or backward until desired angle is reached, optimizing the visual field, balance, function, comfort, and contact with the individual. Once the proper angle is determined, tighten the screws.

Hints:

If adjusting the angle of the back while the individual is in the wheelchair, make sure to support the back when loosening screws to prevent abrupt movement of the back.

If the user has a pronounced kyphosis, neck fatigue and a decreased visual field may be experienced throughout the day while the individual is seated in the wheelchair. Ensure the Jay Care Back angle and seat depth are adjusted to improve visual field while maximizing the individual's function and maintaining comfort.

If additional angle is desired (a maximum of 39° recline is possible), the bottom pins can be adjusted backwards relative to the shell. This will cause the bottom of the back to move forward. However, this will result in a loss of seat depth. If maximum angle is used, assess wheelchair for anti-tip bars.

As a general rule, keep the Back in the most vertical position possible to still allow the user's hips to be kept to the back of the wheelchair and his or her eye level to be parallel with the floor. Over-reclining can accentuate sliding of the hips and hyper-extension of the neck forward.

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EASY CLEANING AND RE-ASSEMBLY FOR JAY CARE BACK**To clean back cover**

Pull foam insert away from back shell. Unzip, remove foam and turn cover inside out, zip closed to avoid snagging. If applicable, note where lumbar shims are placed to ensure proper re-assembly. Machine wash in warm water. Dry in dryer on medium heat. The Quick-Dry cover is made of a 100% synthetic suede that "wicks" moisture away from the skin, allowing the fabric to dry at an accelerated rate.

Note: Do not dry clean covers or use industrial washers and dryers to clean cover. Do not steam autoclave. Do not bleach.

Replace foam in cover and zip closed. The "Top" label on the cover should be at the top, rear of the foam. Ensure that the cover fits properly by aligning the seams with the edge of the foam.

Replace lumbar shims if applicable. Place foam insert on shell. Ensure that "Top" label on cover is at the top of the back shell. Press into the back shell, matching the Velcro® strips on the cover with the Velcro® strips on the shell.

Hint:

In an institutional setting, write the client's name on the content label of the back cover and complete the enclosed "Name" sticker and attach to the back shell. This will ensure the return of the cover after washing.

To clean back shell

Shell may be wiped clean with soap and water. No other maintenance should be required.

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ENGLISH**24 MONTH LIMITED WARRANTY**

Each Jay Care Back is carefully inspected and tested to provide peak performance. Every Jay Care Back is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the back's removable cover. Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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**ENGLISH**

Sunrise Medical
North American Headquarters
 2842 Business Park Avenue
 Fresno, CA, 93727, USA
 (800) 333-4000
 (800) 300-7502
www.SunriseMedical.com

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As the manufacturer, SUNRISE MEDICAL,
 declares that this product conforms to the
 Medical Device Regulation (2017/745).

Manufactured under one or more of the following U.S. patents, plus U.S. and foreign patents pending.
 U.S. Patent #: 5,062,677; 5,149,173; 5,211,446; 5,352,023; 5,407,248; 5,524,971; 5,556,168; and foreign patents

© 2020, Sunrise Medical 6.98 Rev.C
 XT3505E

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Sunrise Medical S.r.l.
Sunrise Medical GmbH