Rumba

Gebrauchsanweisung
Directions for use
Notice D'Utilisation
Gebruikershandleiding
Manuale Per l'Uso
Manual De Utilizaçao
Manual De Usuario

QUICKIE®

Fig. 2

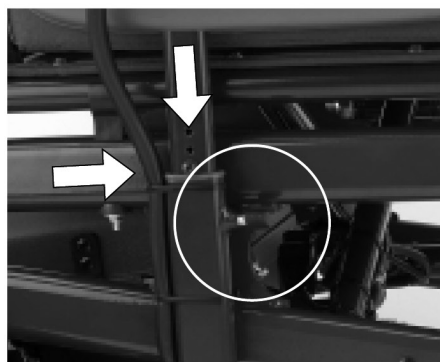
Fig. 3

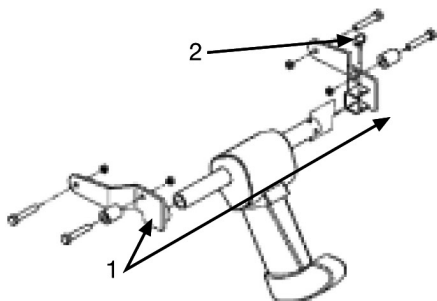
Fig. 4

Fig. 5

Fig. 6

Fig. 7

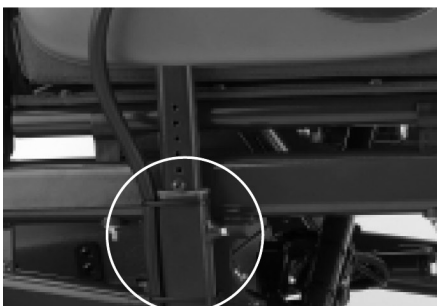
Fig. 8


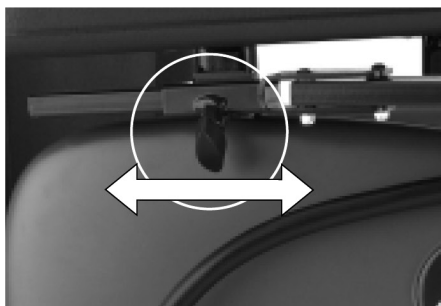
Fig. 9

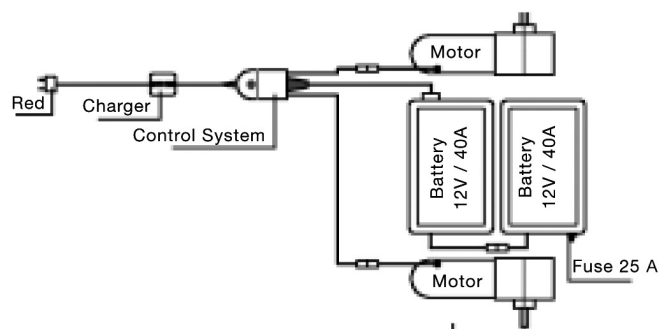Verkabelungsschema
Wiring Diagram
Schéma du câblage
Bekabelingsschema
Schema del cablaggio
Esquema da cablagem
Esquema del cableado

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Intended use power wheel chairs:

Use

Power wheelchairs are exclusively for a user who is unable to walk or has limited mobility, for their own personal use in- and outdoor.

When an Attendant Control Module is fitted, the Power Wheelchair may be operated by an assistant on behalf of the user.

When a Dual Control Module is fitted the Power Wheelchair may be operated by the user, or control may be switched to an assistant to operate on behalf of the user.

The maximum weight limit (includes both the user and any weight of accessories fitted to the wheelchair) is marked on the serial number label, which is affixed to the chassis of the chair.

Warranty can only be taken on if the product is used under the specified conditions and for the intended purposes.

The intended lifetime of the wheelchair is 5 years. Please DO NOT use or fit any 3rd party components to the wheelchair unless they are officially approved by Sunrise Medical.

Area of application

The variety of fitting variants as well as the modular design mean that it can be used by those who cannot walk or have limited mobility e.g. because of:

- Paralysis
- Loss of extremity (leg amputation)
- Extremity defect deformity
- Joint contractures/joint injuries
- Strokes and brain injuries
- Neurological disabilities (e.g. MS, Parkinson...)
- Illnesses such as heart and circulation deficiencies, disturbance of equilibrium or cachexia as well as for elderly people who still have the strength in the upper body.
- Persons who are mentally and physically able to control an input device to operate the chair and its functions in a safe way.

When considering provision, please also note the body size, weight including the distribution of body weight, the user's physical and psychological constitution, the age of the user, their living conditions and their environment. If in doubt a health care professional should be involved to ensure the user is not exposed to unacceptable risks.

Sunrise Medical is ISO 13485 certified, which ensures quality at all stages of the development and production of this wheelchair.

**IMPORTANT:**

DO NOT USE YOUR WHEELCHAIR UNTIL THIS MANUAL HAS BEEN READ AND UNDERSTOOD. NOT FOLLOWING THESE INSTRUCTIONS MAY RESULT IN PHYSICAL INJURY, DAMAGE TO THE PRODUCT OR DAMAGE TO THE ENVIRONMENT!

As the manufacturer, SUNRISE MEDICAL, declares that this product conforms to the Medical Device Regulation (2017/745).

Sunrise Medical declares that this product fulfils the performance requirements for a "Crash Test" to ISO 7176-19.

This symbol means Medical Device

Notice to the user and/or patient: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

B4Me special adaptations

Sunrise Medical strongly recommends that in order to ensure that your B4Me product operates, and performs as intended by the manufacturer; all the user information supplied with your B4Me product is read and understood, before the product is first used.

Sunrise Medical also recommends that the user information is not discarded after reading it, but it is kept safely stored for future reference.

Medical Device Combinations

It may be possible to combine this Medical device with one or more other Medical Device or other product. Information on which combinations are possible can be found at www.Sunrisemedical.co.uk. All combinations listed have been validated to meet the General Safety and Performance Requirements, section 14.1 of the Medical Device Regulation 2017/745.

Guidance on the combination, such as mounting, can be found at www.SunriseMedical.co.uk.

INTRODUCTION

Thank you for choosing a SUNRISE MEDICAL product. In view of the broad range of components and accessories available for all SUNRISE MEDICAL wheelchairs, it is in your interest to consult a health professional or authorised SUNRISE MEDICAL distributor to be sure that your wheelchair is properly adjusted for your safety and comfort. It is advisable to read this instruction manual, especially the safety advice and the sections regarding adjustment, before using the wheelchair.

The QUICKIE RUMBA wheelchairs are designed for persons with a physical handicap, to be used only for transport. These wheelchairs are suitable for use both indoors and outdoors (Class B product).

The maximum user weight is limited at 125 kg and the maximum incline rated at 10° (18%).

Accessories:

- Kerb climber
- Height-adjustable footrests
- User control
- Companion control

If you have any questions regarding use, maintenance, or safety of your SUNRISE MEDICAL wheelchair, please consult one of our authorised distributors:

Note: The user of this product should be examined by a health professional so that it can be verified that the appropriate model has been prescribed with the appropriate accessories. The user should be advised by a health inspector so that he may be sure his own limitations and physical condition do not hinder the safe use of a wheelchair in the course of daily life.

Clean the frame regularly with a non-abrasive cleaner, subsequently polished with non-abrasive wax. You can clean off small stains with a sponge moistened in gentle detergent and warm water. Rinse with water. Do not put upholstery in a dryer.

Sunrise Medical
Thorns Road
Brierley Hill
West Midlands
DY5 2LD
England
Phone: 0845 605 66 88
Fax: 0845 605 66 89
www.SunriseMedical.co.uk

Dealer stamp

Sunrise Medical is ISO 13485 certified, which ensures quality at all stages of the development, and production of this wheelchair. **This product complies with the Medical Device Regulation (EU) 2017/745.**


A. SAFETY

If you follow the instructions given in this manual, you will enjoy many trouble-free years of use: This vehicle is not designed for use on the road except for crossing from one kerb to another. Load data always refers to one single person as operator. The wheelchair is authorised for use by one person.

1. Preliminary checks: You should not try to manoeuvre at full speed. If you have to make a sudden turn, slow down first using the joystick or the speed control knob. This is very important if you are going down a hill. Lack of attention to this point may result in your overturning the wheelchair. Always check that the wheelchair is switched off before trying to get in or out. Always check that you can operate all the controls from a comfortable seated position. It is essential to pay attention to posture in order to assure comfort and well-being. Always check that the freewheel device is switched off before setting the chair in motion. Always check that you can be seen clearly, especially if you are using the wheelchair in conditions of low visibility.

2. Emergency braking: If power is cut by means of the ON/OFF button, braking will be immediate and total. This method of stopping is not recommended except for emergencies, since the stop will be extremely sudden.

3. Kerbs: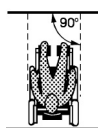

- ☐ Give due attention to other road users. Remember that the last thing a car or truck driver expects to see is a wheelchair descending onto the road from a kerb. If you are at all doubtful, do not cross the road until you have checked that there is nothing coming.
- ☐ Always cross the road as rapidly as possible, since other vehicles may appear.
- ☐ Do not attempt to climb or to descend from, a kerb of more than 100 mm. height.
- ☐ Do not attempt to ascend high kerbs, steep slopes or cambers due to the risk of falling out of the chair or tipping over.
- ☐ Do not attempt to ascend any kerb in the vicinity of a drain cover, uneven surfaces, or gravel covering.
- ☐ Do not attempt to descend a kerb of more than 50 mm in forward position.
- ☐ Do not attempt to climb or descend any kerb at an oblique angle. Approach only at an angle of 90° with at least 500 mm run-up.

4. Tyres: The tyres of a wheelchair suffer wear depending on how much the chair is used. Check them regularly, especially for inflation pressure, according to the service instructions in this manual.

Caution: NEVER inflate the tyres with the air hose of a service station.

5. Freewheel device: Only use this function to push the chair manually from one spot to another. Keep in mind that while the freewheel system is in operation, the chair is without a braking system. Disengaging the motor causes an error message to appear in the control, which is indicated when the LED display flashes rapidly. For safety reasons the wheelchair can then no longer be moved using the joystick.

6. Transportation in vehicles

This product is suitable to use as a seat in vehicles as per certificate **ISO / NP 7176-19: 2001**, and using the restraint system Unwin (**4 point WWR / ATF / K / R**)

A wheelchair secured in a vehicle will not provide the equivalent level of safety and security of a vehicle seating system. It is always recommended that the user transfers to the vehicle seating. It is recognised that this is not always practical for the user to be transferred and in these circumstances where the user must be transported whilst in the wheelchair the following advice should be followed.

1. Confirm that the vehicle is suitably equipped to transport a passenger in a wheelchair, and ensure the method of access/egress is suitable for your wheelchair type. The vehicle should have the floor strength to take the combined weight of the user, the wheel chair and accessories.
 2. Sufficient space should be available around the wheelchair to enable clear access to attach, tighten and release the wheelchair and occupant tie down restraints and safety belts.
 3. The occupied wheelchair must be located in a forward facing position and secured by the wheelchair tie down and occupant restraint straps (WTORS tie downs) meeting the requirements of ISO 10542 or SAE J2249, in accordance with the WTORS manufacturers' instructions.
 4. The wheelchair's use in other positions within a vehicle has not been tested e.g. transportation in a side facing position must not be carried out under any circumstances. (Fig. 10)
- **Warning** There is a risk of serious injury or death if this advice is ignored.

5. The wheelchair should be secured by a Tie Down Restraint system, conforming to ISO 10542 or SAE J2249 with non-adjustable front straps and adjustable rear straps, which typically use Karabiner clips/S hooks and tongue and buckle fittings. These restraints generally comprise of 4 individual straps that are attached to each corner of the wheelchair.
6. The tie-down restraints should be fitted to the main frame of the wheelchair as indicated in the diagram on the following page, and not to any attachments or accessories, e.g. not around the spokes of wheels, brakes or footrests.
7. The tie-down restraints should be attached as close as possible at an angle of 45 degrees and tightened securely in accordance with the manufacturer's instructions.

8. Alterations or substitutions must not be made to the wheelchair securement points or to structural and frame or components without consulting the manufacturer. Failure to do so will invalidate the ability of a Sunrise Medical wheelchair to be transported within a vehicle.
9. Both pelvic and upper torso restraint belts must be used to restrain the occupant to reduce the possibility of head and chest impacts with the vehicle components and serious risk of injury to the user and other vehicle occupants. (Fig. 11) The upper torso restraint belt should be mounted to the vehicle "B" pillar - failure to do so will increase the risk of serious abdominal injuries to the user.

Fig. 10


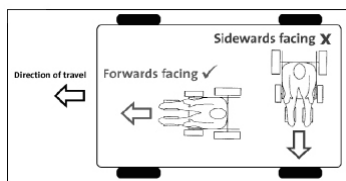
Fig. 11

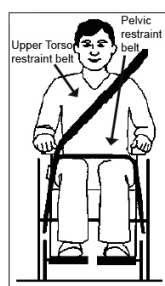
10. A head restraint suitable for transportation (see label on headrest) must be fitted and suitably positioned at all times during transportation.
11. Postural supports (lap straps, lap belts) should not be used or relied on for occupant restraint in a moving vehicle unless they are labelled as meeting the requirements specified in ISO / NP 7176-19: 2001 or SAE J2249.
12. The safety of the user during transportation depends upon the diligence of the person securing the tie-down restraints and they should have received appropriate instructions and/or training in their use.
13. Wherever possible remove and stow safely away from the wheelchair all auxiliary equipment, for example:
Crutches
Loose cushions
Tray Tables
14. Articulating/elevating leg rest should not be used in the elevated position when the wheelchair and user are being transported and the wheelchair is restrained using Wheelchair Transport and Occupant Restraints.
15. Reclining backrests should be returned to an upright position.
16. The manual brakes must be firmly applied.

Occupant Restraint Instruction

1. The pelvic restraint belt must be worn low across the front of the pelvis so that the angle of the pelvic belt is within the preferred zone of 30 to 75 degrees to the horizontal. A steeper (greater) angle within the preferred zone is desirable i.e. closer to, but never exceeding 75degrees. (Fig 12)

Fig. 12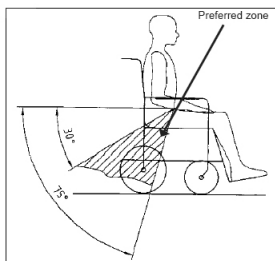

- The upper torso restraint belt must fit over the shoulder and across the chest as illustrated Fig 13 and Fig 14.

Restraint belts must be adjusted as tightly as possible consistent with user comfort.

Restraint belt webbing must not be twisted when in use.

The upper torso restraint belt must fit over the shoulder and across the chest as illustrated in Fig 13 and Fig 14.

- Restraints should be mounted to the vehicle "B" pillar and should not be held away from the body by wheelchair components such as armrest or wheels.

Fig. 13


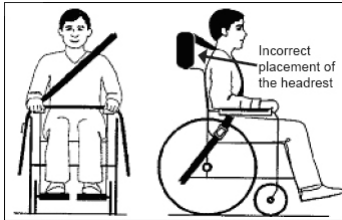
Fig. 14

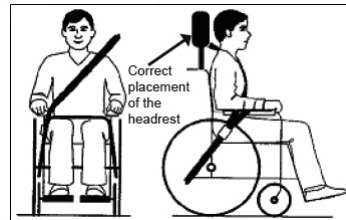- The attachment points to the chair are the inner front side frame just above the castor and the rear side frame. The straps are fitted around the side frames at the intersection of the horizontal and vertical frame tubes. (See Figs 15-20)
- The tie down symbol (Fig 16) on the wheelchair frame indicates the position of the wheelchair restraint straps. The straps are then tensioned after the front straps have been fitted to secure the wheelchair.

Transportability – positioning of wheelchair tie down restraints on wheelchair

Fig. 15

Fig. 16 Tie down label

The wheelchair secured with front and rear wheelchair tie down restraints. Positioning of the restraint straps is shown in more detail below.

Fig. 17

Fig. 18

Position of the front wheelchair tie down restraint and the tie down label.

Fig. 19

Fig. 20

Position of the rear wheelchair tie down restraint and the tie down label on the wheelchair.

7. Mobile telephones: Mobile phones should never be used while the wheelchair is being driven, since they may give rise to excessively strong electromagnetic fields that could interfere with the electronic systems of the wheelchair. If you have to use your mobile, stop the chair first and switch off. Keep away from aerials, antennas and high voltage electricity pylons. Take into account that the wheelchair itself can disturb electromagnetic fields such as emitted by alarm systems of shops.

8. Weather: Contact with water, snow or ice may affect the performance of your wheelchair. Never use your wheelchair in a pool or other body of water. If your chair becomes wet, or if you use water to clean it, dry the chair thoroughly as soon as you can.

9. Hot surfaces: Not only motors can become hot during the operation of the chair, the upholstery material and armrests may heat up when standing in the sun.

10. Lap belt:

Before using your wheelchair ensure the seat belt is worn and correctly adjusted before use.

The lap belt is fitted to the wheelchair as shown in the fitting instructions section c.7. It is important that the belt is secured around the back of the wheelchair. The straps can be fed between the armrest and backrest.

Adjust the belt position so that the buckles are in the centre of the seat. See fig 4 of fitting instructions section c.7

Adjust the lap belt to suit the user's needs as follows:

To increase the belt length: **To reduce the belt length:**

Feed the belt through slide adjusters and male buckle to provide more belt length.

Feed the belt back through male buckle and slide adjusters.


Ensure the belt is not looped at the male buckle


When fastened check space between the lap belt and the user, when correctly adjusted it should be possible to insert the flat of the hand between the lap belt and the user.

Generally the Lap Belt should be fixed so that the straps sit at an angle of approximately 45°, and when correctly adjusted should not allow user to slip down in the seat.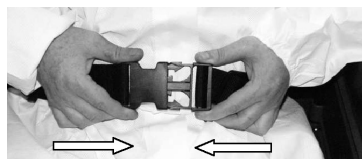

To fasten buckle:
Firmly push the male buckle into female buckle.

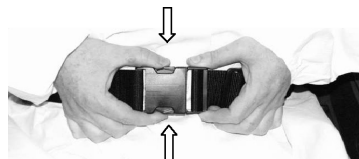To release belt:
Press the exposed sides of the male buckle and push towards the centre whilst gently pulling apart.

Do not rely on the lap belt only when the wheelchair and occupant are transported in a vehicle, use the separate occupant lap and diagonal restraints provided in the vehicle.

Advice to Client

The lap belt must be checked on a daily basis to ensure it is adjusted correctly and it is free from any obstruction or adverse wear.

Failure to make sure that the lap belt is secure and adjusted prior to use could cause serious injury to the user. E.g. too loose a strap may allow the user to slip down in the chair and risk suffocation.

Maintenance

Check lap belt, and securing components; at regular intervals for any sign of fraying or damage. Replace if necessary.

Clean the lap belt with warm soapy water and allow to dry.

Note

The lap belt should be adjusted to suit the end user as detailed above

Sunrise Medical also recommend that the length and fit of the belt is checked on a regular basis to reduce the risk of the end user inadvertently re-adjusting the belt to an excessive length

If in doubt about the use and operation of the seat belt then ask your healthcare professional, wheelchair dealer, carer or attendant for assistance.

NOTES:

- The rider plus item should never exceed a total weight of 125 kg.
- Before you use the chair, carefully check the functionality.
- If you discover any chair malfunction, take it to be repaired or reset. Your dealer can help you to find the fault and correct it.
- Make sure batteries are charged.
- Be aware that there is a point during every transfer when the wheelchair seat is not below you.
- Use extra care when you drive your chair in reverse. If one of the wheels hits an obstacle, you could lose control of the chair or fall out.
- Never use your chair on a slope unless you are sure you can do so without losing traction.
- Lift the chair only by non-detachable parts of the main frame.
- Never short-circuit electrical connections as you could cause an explosion.
- Do not use this chair if any of the tyres are under or over-inflated.
- When using mobile phones you should switch off the chair to avoid causing electromagnetic radiation.

B. CHARACTERISTICS (Fig. 1)

Back upholstery	1	Motors.....	8
Armrest.....	2	Castors	9
Control lever.....	3	Footrest	10
Height and width-adjustable armrest.....	4	Cushion	11
Stabilising bar.....	5	Control position	12
Battery boxes	6	Joystick.....	13
Drive wheel.....	7		

C. ASSEMBLY

1. Footrest (Fig. 2): The footrests can be swung away by moving the release lever in the swing-away direction. To remove the footrest, activate the release lever, rotate the footrest approx. 90° outwards and then pull it out upwards. To refit the footrest follow the reverse procedure and the lock engages automatically when swung inwards.

2. Armrest (Fig. 3): Slide the armrest tube into the receiver of the wheelchair and turn the lever to the right to secure it. If you require a different individually fixed armrest height, you can set this using the adjusting screw on the connecting tube.

3. Anti-tip wheels: These are mounted when the wheelchair leaves the factory. The anti-tip wheel tube is screwed firmly to the motor retaining plate of the wheelchair frame. The anti-tip wheels can be removed by slackening and unscrewing the two M8 Allen screws. Reverse the procedure to assemble. Note: If anti-tips wheels are fitted, the wheelchair may interfere with kerbs when mounting or dismounting.

4. Kerb climber (Fig. 4):

- The left and right mounting plates for the Kerb Climber Tube (1) are fixed onto the inside of the frame using the sleeves, nuts and bolts provided.
- Next fit the Kerb Climber Tube, starting on the left hand side and fitting the end of the tube into the round socket on the left-hand mounting plate.
- Then fit the right-hand end into the square socket on the right mounting plate and pass the quick release axle through it (2).

5. Batteries

1. Lean the side supports of the battery housings on the rails.
 2. Push the battery cases forwards on the rails until they lock in place. Put the battery case with the connectors for the electronics in first. Position it first so that the connecting lead for the electronics is on your right. This will make it much easier to connect the batteries.
 3. Now connect the two connecting plugs on the left side of the battery cases together.
 4. Once you have made sure that the connections are secure, connect the front battery to the controls.
- To remove the batteries, reverse the procedure. Disconnect the battery from the control and then disconnect the batteries from each other. Push the battery cases upwards slightly until they disengage, and then pull them out along the rails.

6. Wheels: If it is necessary to remove the wheels, to repair a damaged tyre for example, proceed as follows:

Front wheel:

1. Use a 7 mm. Allen wrench to remove the wheel axle from the fork.
2. Remove the damaged wheel.
3. When the wheel has been repaired, follow the reverse procedure to refit.

Rear wheel:

1. Using a screwdriver, remove the cover of the hub.
2. Loosen this screw with a 19 mm. hexagonal socket wrench.
3. Remove the damaged wheel. When it has been repaired, follow the reverse procedure.

NOTE: The rear wheels are designed as separate rims and can be disassembled to simplify changing the tube or cover.

7. Lap Belt

- 1) Remove Lap belt (pt No. 52596) from Bag
- 2) Undo the backrest lever and remove fixing bolt / hand wheel and washer.

- 3) Using the fixings previously removed; fix the lap belt to the back post via the desired hole depending on Back height required.

Ensure the saddle washer is fitted onto the Back tube with the washer fitting between the belt and the hand wheel.

4) Tighten the hand wheel ensuring the belt is fed towards the seat as shown below. 5) Lap Belt fitted to Rumba

D. USE

1. Handling the wheelchair

Folding for transportation: Remove the batteries as described in Chapter C part 5, and ensure that the footplates are flipped up or removed, as described in Chapter C part 1. Take hold of the seat upholstery by the middle and pull it upward until the unit is fully folded. To lift the batteries please use the handles on the lid of the battery boxes. To lift the chair when folded please use the upper parts of the side frame tubes and back handles.

Opening: Press downwards on the upper tubes of the cross frame until the chair is completely unfolded. Now put the batteries in place. Note: Never lift the wheelchair by the armrests, since they are detachable and harm could be done to the user or to the wheelchair.

Reclining back (0° - 30°) (Fig. 5):

The backrest can be adjusted in 7,5° increments by removing the two screws. To fix the required back angle, insert the two bolts again and tighten with the nuts.

Caution: For safety reasons never remove the anti-tip wheels.

Caution: The reclining back is only to be used in combination with a stabilising bar. Please always make sure that the stabilising bar is locked in place on both sides.

Caution: Before folding the wheelchair, make sure that the stabilising bar is removed.

Reclining back:

Caution: The reclining back is only to be used in combination with the active anti-tips setting.

Caution: The reclining back is only to be used in combination with a stabilising bar. Please always make sure that the stabilising bar is locked in place on both sides.

Caution: Before folding the wheelchair, make sure that the stabilising bar is removed.

Caution: In order to avoid any reduction in the cable function, you must make sure that the cables are not completely taut.

Caution: When adjusting the back angle, be careful not to get your fingers caught.

Caution: Before adjusting the back angle, put the anti-tip tubes in to the active position.

Height-adjustable footrests: To elevate: Lift the leg and the swing-up part of the leg support. The leg support engages automatically as soon as you let go of it. To lower: Lift up the leg and press the lever to release while lowering the footrest. This will now remain in its new position as soon as you let go of the lever.

Caution: Keep hands clear of the adjustment mechanism between the frame and the moving parts of the footrest while adjusting the legrest.

To steer the chair simply move the joystick or knob in the desired direction. The further forward you push it, the faster the wheelchair will travel. The knob regulates the power and top speed. When it is released, the brakes will activate automatically and the wheelchair will come to a standstill. It is important that you stop the wheelchair before you change direction from forward to reverse. Always turn off the wheelchair before you get off. Look ahead while you drive, not at the controls. Remember that only with practice will you become a competent driver.

Joystick

Your wheelchair is controlled by:

- ☐ Joystick: This determines the speed and direction of the wheelchair. The further forward you push it, the faster the wheelchair will travel. When the joystick returns to its central position, the brakes will activate automatically. On flat ground, braking distance is equal to the length of the wheelchair. On a hill it may be a little more.

Other controls: All further devices are explained in the appendices:

1. Battery Gauge/fault indicator: This indicates that the wheelchair is under power. It also indicates any fault there may be in functioning. The number of lights flashing indicates the fault type. (See "Fault indicator" section in this manual.)
2. Speed indicator: This indicates the maximum speed set for the wheelchair. There are five predefined speeds, 1 is the lowest and 5 is the highest.
3. On/Off key: This connects or disconnects the entire electronic system, which supplies energy to the motors. Do not use this button to stop the chair except in case of emergency, as doing so may damage the wheelchair.
4. Horn
5. Speed decrease key: Decreases the maximum speed setting
6. Speed increase key: Increases the maximum speed setting

□ Fault indicator:

- If it remains lit, this means that everything is OK. It flashes to indicate any fault that may have arisen. Slow and regular flashing means the batteries need charging.
- If it flashes once every 2.5 seconds, this means that the control system has been deactivated particularly if the wheelchair has not been used for a certain period of time. To reactivate it, turn off the system and then turn it on again.
- If the fault indicator flashes rapidly, this indicates that there is a fault. Turn off the system. Check the state of the batteries and the connections, and turn on the system again. If the flashing continues, the number of bars flashing will indicate the location of the fault.
- If the number of bars flashing is 1: The batteries need charging, or are badly connected.
- If the number of bars flashing is 2: The left motor is improperly connected.
- If the number of bars flashing is 3: Electrical fault in the left motor.
- If the number of bars flashing is 4: The right motor is improperly connected.
- If the number of bars flashing is 5: Electrical fault in the right motor.
- If the number of bars flashing is 6: The battery charger is plugged in.
- If the number of bars flashing is 7: Fault in the control.
- If the number of bars flashing is 8: System fault. Check all connections.
- If the number of bars flashing is 9: Fault in the electric brake.
- If the number of bars flashing is 10: Operating voltage too high.

Electrical fuses: To prevent the wheelchair from becoming overloaded, an electronic safety system has been built in, which automatically disconnects the battery from the control. This device cuts in only in extreme circumstances, such as repeated climbing of steep hills in very high ambient temperatures. The fuse button, located on the side of the battery, trips immediately and the wheelchair comes to a stop. The fuse can be reset by pressing the button. After a few seconds the wheelchair can be driven again normally.

Slopes: The following instructions explain how to manoeuvre correctly on steep ramps and slopes, especially in the course of descent. If you stop on an incline, start off slowly and, if necessary, lean forward to counteract the tendency of the front wheels to rise. On descents it is important not to allow the wheelchair to exceed normal speed. Indeed, the safest course is to go down steep hills slowly and to stop immediately if you have any doubts about the steering. If the chair gathers speed, move the joystick to its central position to slow down, or stop completely. Then start off again slowly and do not allow speed to increase. Make sure the automatic brakes are engaged. Climbing or descending a slope with the automatic brakes disengaged can be dangerous.

Transfers: Always position the wheelchair close to the place where the transfer will be carried out. It is important that the controller is turned OFF, so that the wheelchair will not move throughout the procedure. The wheelchair is highly stable owing to its weight. Do not stand up on the footrests at any time during the transfer.

Freewheel device (Fig. 6):

Turn the lever on the back of each motor outwards so that each drive wheel is in freewheel position and will turn freely. If you wish to push the wheelchair with the motors off, you should follow this procedure with both wheels. When the levers are turned inwards, the drive wheel will automatically connect with the propulsion mechanism. Follow this procedure with both wheels.

2. Batteries:

24 V (2x12 V) / 50 Amph., maintenance-free

Dimensions: 197 x 165 x 170 mm

Capacity at the discharge time 34 Ah/5 h

Connector: 3 pin "Cannon" type

Charger: 24 VDC 8 Amph

Charging batteries: Charger Specification: Only use chargers for cyclic batteries, Gel-Type with 24V/8A output nominal max. To charge batteries, connect the charger to the control and then to the 220V50Hz mains. For more information about its operation consult the instructions provided with the charger. If the charger is not connected to the control it does not provide charge current. It is possible that the charger's metallic box may slightly raise its temperature due to use.

General care of batteries:

- ☐ Avoid heavy discharge of the batteries. If you continue driving until the chair is virtually immobile, you will have reduced the useful life of the batteries.
- ☐ The batteries should never be left uncharged. A battery that remains uncharged for two weeks loses a certain capacity that cannot be recovered later.
- ☐ The batteries should always be charged completely. The charging process should not be interrupted.
- ☐ Your batteries should be charged only by means of the charger supplied with the wheelchair. The battery needs a "running-in" period over six/twelve charges; and will not attain to its maximum range until the end of this period. It is better to limit the use of the wheelchair and the length of your trips until the batteries have been totally "run in".
- ☐ It is advisable to charge batteries every day, after using the wheelchair.
- ☐ Range: Batteries give the chair a range of approximately 25 kilometres. This depends on the weight of the user and the terrain.
- ☐ Do not leave the batteries exposed to environmental conditions such as rain, humidity, high temperatures or fire.
- ☐ Check the conditions of the batteries and their connections to prevent possible dangers e.g. leaks.

Safety guidelines for charging

- ☐ The charger is designed for indoor use only. Do not use outdoors or expose to rain, snow, spray or moisture.
- ☐ Suitable temperature for charging the batteries: -10°C up to +40°C.
- ☐ The use of an accessory that has not been sold by Sunrise Medical, or that does not have Sunrise Medical's approval, may result in the risk of fire, electrocution, or personal injury.
- ☐ Make sure the cable is laid out in such manner that it cannot be stepped on, that it cannot be tripped over, and that it is not exposed to damage or stress.
- ☐ Extension cables should not be used unless there is no alternative. The use of the wrong extension cable could cause fire or electrocution. If an extension cable must be used, make sure the pins of its plug are equal in number, size, and shape to those of the charger plug, that the cable is correctly connected.
- ☐ Do not use the charger if it has received a hard knock, been dropped, or if it has been otherwise damaged. Take it to a qualified technician.
- ☐ Do not disassemble the charger. For repair it should always be handed over to the manufacturer. The wrong installation could lead to fire or electrocution.
- ☐ Never place the charger directly on top of the battery being charged.
- ☐ Never leave the charger on a rug or soft surface. Always place it on a hard surface.
- ☐ Due to an electronic circuit of the charger, the batteries can be permanently connected without any harm.

IMPORTANT NOTES:

- ❑ Never smoke, and never allow the generation of sparks or flames, in the vicinity of battery or charger.
- ❑ Take special care to ensure that metal tools cannot fall on the battery. A spark or short circuit could result in the battery or other electrical component that would cause an explosion. Before working on the battery, remove all personal effects that are metal and loose fittings from your person.
- ❑ When buying spare batteries, only use types which are approved by Sunrise Medical. If in doubt, consult your Sunrise Medical service agent.

3. Tyre pressure: It is important to check inflation pressure regularly, along with the state of wear of the tyres. The maximum pressure recommended for the front wheels is 36 psi (2.5 bars), while for the rear wheels it is 40 psi (2.7 bars). Caution: never inflate the tyres with a service station air pump. It is recommended that you use a manual pump or a pressure regulator (manometer).

4. Kerb climberClimbing kerbs

1. Go forward to the kerb at a low and uniform speed.
2. When the kerb climber meets the kerb, the wheelchair should be moving slowly. Low kerbs can be climbed from a stationary position.
3. Allow the motors sufficient power to raise the front of the chair so that it will be supported on the kerb. Then release a little more power and increase speed so that the driving wheels climb the kerb smoothly and without interruption. As far as possible, hold the joystick in forward-drive position.

Descending from kerbs

1. Approach slowly and carefully in reverse until the rear wheels are at the edge.
2. As slowly as possible, descend from the kerb in reverse and on the rear wheels. You will feel more secure if you lean forward, but do not worry if you are unable to: the wheelchair is extremely stable. The wheelchair is extremely stable and as long as you do not exceed the construction limits, there is no risk.
3. The front part of the chair will in turn descend naturally from the kerb as the chair continues away from it in reverse.

It may help, in a difficult situation, to lean back while the front ascends, and lean forward while the rear wheels rise. However, too much movement in the seat at the wrong moment may make the situation worse, for which reason it is better not to change position more than necessary.

E. ADJUSTMENTS

1. Adjustment of the footrest (Fig. 7): To adjust the height of a footrest, remove the securing nut from the screw in the tube. Remove the screw, set the footrest at the height desired, and replace the screw in the hole corresponding to that height. Secure the screw with the nut. For these adjustments it is recommended you use a 10 mm. spanner.

Note: Make sure that the screws are tightened securely.

2. Adjustment of the armrests:

Height adjustment (Fig. 3): Set the screw in the hole of the armrest tube that corresponds with the desired height. Turn the handle to the right to avoid possible play

Seat width adjustment (Fig. 8): To adjust the width between the armrests, use a size 10 mm hexagonal wrench to loosen the hexagonal screws on the armrest adapter retainers, which are located on the side frame below the armrests, and pull the armrests outwards or inwards to achieve the desired position. Tighten the screws securely. Make sure that the screws are secured with the counter-nuts.

3. Control Adjustment (Fig. 9): The control can be adjusted to various positions by pushing it along the armrest. The control can be moved by turning the lever below the arm padding to the left. The position can be secured by turning the lever to the right.

4. When the chair leaves the factory, the parameters of the **controller** are set to default. However, when you want to carry out any alterations, then you should either send the wheelchair to Sunrise Medical or contact your dealer, where a specialist can carry out this work. In this way incorrect programming and therefore incorrect function of the wheelchair can be avoided.

Note: A programmer or a PC-based software program with the appropriate connecting lead are required to program the wheelchair. Programming the controller of the wheelchair is only allowed through authorised Sunrise Medical agents. Wrong controller settings could cause driving outside the safe limits and could cause damage and injury.

F. CONTROLS AND MAINTENANCE

	Daily	Weekly	Monthly	Quarterly	Annually
Charge battery	X				
Check battery level indicator	X				
Ensure all removable parts are securely fastened	X				
Check all the electrical connections		X			
Wipe vehicle with a damp cloth		X			
Check tyre pressures		X			
Chek tyres for wear			X		
Clean and protect the battery terminals with petroleum jelly				X	
Check condition of upholstery				X	
Complete inspection and service by an authorised supplier					X

G. SPECIFICATIONS

	Min.	Max.		Min.	Max.
Effective seat width	420 mm.	540 mm.	Seat plane angle	3°	
Seat depth	430 mm.		Backrest angle	0°	30°
Backrest height	430 mm.	490 mm.	Leg to seat surface angle	70°	
Overall length (w/legrest)	1070 mm.	1100 mm.	Max. Speed forward		6 km/h.
Overall width	570 mm.	630 mm.	Obstacle climbing (w/kerb climb.)	50 mm.	100 mm.
Climbing ability upwards		10° (18%)	Max. safe slope		10°
Seat surface height at front edge	500 mm.		Min. Turning radius	850 mm.	
Footrest to seat distance	400 mm.	470 mm.	Max. User weight		125 kg.
Armrest to seat distance	180 mm.	280 mm.	Mass of the heaviest part (frame without detachable parts and batteries)		37,4 kg.
Folded length	1070 mm	1100 mm.			
Folded width	400 mm.				
Folded height	940 mm.				
Total mass (w/batteries)	75 kg.				

H. GUARANTEE

THIS GUARANTEE DOES NOT AFFECT YOUR LEGAL RIGHTS IN ANY WAY.

Sunrise Medical* provides a guarantee, as set out in the warranty conditions, for wheelchairs to its customers covering the following.

Warranty conditions:

1. Should a part or parts of the wheelchair require repair or replacement as a result of a manufacturing and/or material fault within 24 months or for frame and cross-braces within 5 years after delivery to the customer, then the affected part or parts will be repaired or replaced free of charge. The warranty will only cover manufacturing defects
2. To enforce the warranty, please contact Sunrise Medical Customer Service with the exact details of the nature of the difficulty. Should you be using the wheelchair outside the area covered by the Sunrise Medical customer service agent, repairs or replacement will be carried out by another agency as designated by the manufacturer. The wheelchair must be repaired by a Sunrise Medical designated Customer Service agent, (dealer).
3. For parts, which have been repaired or exchanged within the scope of this warranty, we provide a warranty in accordance with these warranty conditions for the remaining warranty period for the wheelchair in accordance with point 1).
4. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, (following the fitting), in accordance with these warranty conditions.
5. Claims from this warranty shall not arise, if a repair or replacement of a wheelchair or a part is required for the following reasons:
 - a. Normal wear and tear, which include batteries, armrest pads, upholstery, tyres, brakes shoes, etc.
 - b. Any overloading of the product, please check the EC label for maximum user weight.
 - c. The product or part has not been maintained or serviced in accordance with the manufacturer's recommendations as shown in the user instructions and/or the service instructions.
 - d. Accessories have been used which are not specified as original accessories.
 - e. The wheelchair or part having been damaged by neglect, accident or improper use.
 - f. Changes/modifications have been made to the wheelchair or parts, which deviate from the manufacturer's specifications.
 - g. Repairs have been carried out, before our Customer Service has been informed of the circumstances.
6. This guarantee is subject to the law of the country in which the product was purchased from Sunrise Medical*

* Means the Sunrise Medical facility from which the product was purchased.

TAKE CARE OF THE ENVIRONMENT: FOLLOW YOUR TOWN'S COUNCIL LAWS TO GET RID OF YOUR USED BATTERIES, CHARGER, ELECTRONIC COMPONENTS AND WHEELCHAIR.

I. SERVICE HISTORY

This section is designed to assist you in keeping a record of any service and repairs to your wheelchair. The service Agent will complete this section and return the manual to you:

Model					Serial No.				
Year	1	2	3	4	Year	1	2	3	4
Service dates					Service dates				
Controller					Chassis				
On/Off switch					Condition				
Output plug					Steering				
Operation					Upholstery				
Dynamic braking					Seat				
Programmable configuration					Back				
Batteries					Armrests				
Levels					Electric's				
Connections					Condition of loom				
Discharge tests					Connections				
Wheels/Tyres					Test run				
Wear					Forward				
Pressure					Reverse				
Bearings					Emergency stop				
Wheel nuts					Left run				
Motors					Right run				
Wiring					Up/down slope				
Noises					Over obstacle				
Connections					Parking brake				
Brakes									
Brushes									

Apart from the chapters on repairs, the service history and the spare parts catalogue, there is no service manual available. Replacement parts are available on request by the dealer for repairing or changing.

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Sunrise Medical B.V.
Sunrise Medical HCM B.V.
Sunrise Medical S.A.S